**The Walker
La marchette
Andador**

**Safety Guidelines
Consignes de
sécurité
Pautas de seguridad**

Cautions

To ensure the safe assembly and operation of this product, these instructions must be followed. **FAILURE TO FOLLOW INSTRUCTIONS MAY RESULT IN SERIOUS INJURY TO THE USER OR DAMAGE TO THE PRODUCT.** If you are unable to understand these warnings, cautions and/or instructions, contact a healthcare professional, dealer or Medline before attempting to use this product.

DO NOT assemble and use this product without first reading and understanding this user's guide, otherwise injury or damage may occur.

Before each use make sure:

Walker is adjusted to the correct height.

Locking mechanisms and spring buttons are fully engaged

Tips are in good condition. Replace tips immediately if worn or missing.

Walker sits level and all legs are adjusted to an equal height.

The product is only used as instructed by your healthcare provider.

- » Engage the lock and practice bending, reaching and transfers on and off the walker.
- » **NEVER** attempt a maneuver that has not been practiced.
- » Be aware of your surroundings when operating the walker.
- » Look for hazards and avoid them.
- » **DO NOT** perform any adjustments while the walker is in use.
- » **DO NOT** operate on loose gravel or uneven surfaces.
- » **DO NOT** travel over large cracks or fissures.
- » Stop and move forward slowly when encountering cracks or separations in walkway surfaces.
- » **DO NOT** hang anything from the walker handles or frame at any time other than a Medline provided basket or bag. This may cause the walker to tip.
- » **DO NOT** put more than 3 lb (1.36 kg) in the bag or basket.
- » Use Medline accessories only. Use of other manufacturer's accessories can result in improper fit that could result in fall risk and/or injury.
- » Use wheels on the front legs only. Wheels can be used on all four legs as long as the appropriate rear wheel brakes are installed.
- » **DO NOT** use on stairs or escalators.
- » Ensure all legs are at the same height adjustment level before use.
- » **DO NOT** exceed the maximum weight capacity listed on the device and in the assembly guide or serious injury could result.
- » **DO NOT** use product for anything other than its intended use.
- » **DO NOT** use if components are missing or are damaged. Contact Medline at **1-800-MEDLINE (633-5463) | 1-800-268-2848**.
- » Ensure walker is completely unfolded before use. Failure to do so may result in possible injury or product damage.
- » Ensure all push buttons are fully protruded and are properly engaged.
- » Ensure the tips are not worn, broken, or missing. Replace any missing or damaged tips before use.

Warranty

Your Medline Product is warranted to be free of material defects in materials and workmanship for the lifetime of the consumer. This device was built to precise standards and carefully inspected prior to shipment. This Lifetime Limited Warranty is an expression of our confidence in the material and workmanship of our products and assures years of convenient, dependable service. This warranty does not extend to non-durable components such as handgrips, pushbuttons, leg riggings and any other wearable part which carry a one (1) year warranty. In the event of a defect or failure covered by this warranty, contact us at **1-800-MEDLINE**. Medline will, at its option, repair or replace your Medline product.

This warranty does not extend to damage or failure occurring as a result of owner misuse or negligence, normal wear and tear, or other causes unrelated to the manufacture and design of the product. This express warranty is Medline's sole and exclusive warranty and your sole remedy in the event of default or failure. Medline is not responsible for any consequential or incidental damages or costs arising from use of the warranted products, or any defect therein or failure thereof. Medline disclaims all other warranties including but not limited to the implied warranties of merchantability and fitness for a particular purpose.

Contact us at **1-800-MEDLINE**, Monday – Friday 8:00 am to 5:00 pm CST if you encounter any problems or for information regarding repairs or maintenance.

Maintenance

1. Clean the frame/hand grip with a mild soap, detergent or household cleaner. Do not use an organic solvent (Such as acetone, MEK, lacquer thinner, lighter fluid, dry cleaning fluid, gasoline or turpentine) that may dissolve the plastic and cause damage to the product. Thoroughly dry the product **IMMEDIATELY** with a towel.
2. Check the hardware prior to each use to ensure that all components are properly tightened and/or fastened.
3. On a monthly basis or as necessary; check the condition of the frame and ensure that it is free of defects.

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Some products may not be available for sale in Mexico or Canada. We reserve the right to correct any errors that may occur within this brochure. © 2023 Medline Industries, LP. All rights reserved. Medline is a registered trademark of Medline Industries, LP.