



WARNING

Improper handling or use of a damaged Portable Transport Unit can cause death or serious injury. Follow all safety instructions.

SAFETY INSTRUCTIONS

- Do not use the Portable Transport Unit if unit is punctured, torn, frayed or excessively worn.
- Do not lift patients who weigh more than 1000 lbs.
- Use handles only to lift patient.
- Use a minimum of 4 people to lift patient.
- Use proper grasping technique when lifting patient.
- Do not use "cross handle" grasping technique.
- Reinforcement straps must face down.
- Do not contact the unit with sharp objects.
- Do not drag the unit over rough surfaces.
- Do not expose the unit to direct heat sources greater than 200° F.
- Do not machine wash or machine dry the unit.
- Clean the unit with damp cloth and soap, detergent or disinfectant.
- Do not use the unit in conjunction with any type of mechanical lifts.