INSTRUCTION MANUAL

5 Qt. SHARPS CONTAINER

MANUAL DE INSTRUCCIONES

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Drop, don't force sharps into container, forcing could cause puncture.

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BIOHAZARD
BIORIESGO

5 Qt. SHARPS CONTAINER INSTRUCTIONS FOR USE

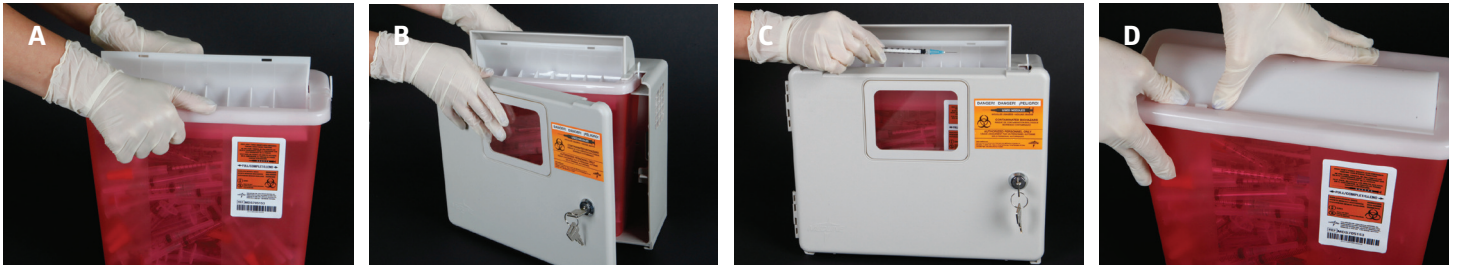
Medline Sharps Containers are intended to provide a receptacle for used, contaminated medical sharps and act as an enclosure during transport to ultimate disposal.

WARNING:

- Once the container is 70% full as indicated by the "full line", no additional contents should be placed in the container.
- Do not attempt to reopen or reuse when full. Dispose of the container properly in accordance with local regulations.
- This container is not intended for direct shipment when full. Place the full container in a secondary transport container for proper sharps disposal in accordance with state and local requirements.

Container Assembly and Instructions for Use:

1. Connect the lid to the base of the sharps container by snapping the grooves of the lid to the top rim of the base. Secure the lid in place by applying pressure to both the front and the back of the lid.
2. Once the lid is secured, place the sharps container in its respective cabinet holder or bracket with the opening of the flap facing you.
3. Drop used sharps into the opening of the lid in a horizontal manner.
4. When the contents of the container meet the indicated full line, you are ready for disposal. To do so, completely pull the two tabs located on the right and left side of the lid in a forward motion.
5. To secure the lid closed, apply pressure to the rounded top and snap the grooves into place. Once the lid is securely locked, it should not be able to be re-opened.
6. Remove sharps container from its respective cabinet holder or bracket and dispose of waste in accordance with local regulations.

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