GUARDIAN[®]

User's Guide

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REF G2-105RX1

General Warnings

To ensure the safe assembly and operation of the product, these instructions must be followed. FAILURE TO FOLLOW INSTRUCTIONS MAY RESULT IN SERIOUS INJURY TO THE USER OR DAMAGE TO THE PRODUCT. If you are unable to understand these warnings, cautions and/or instructions, contact a healthcare professional, dealer or Medline before attempting to use this product.

- DO NOT assemble and use this product without first reading and understanding this user guide, otherwise injury or damage may occur.
- DO NOT use if components are missing or are damaged, contact Medline at 1-800-MEDLINE | 1-800-268-2848.
- DO NOT exceed weight capacity of 350 lbs (158 kg).
- DO NOT make any adjustments while using the product.
- DO NOT attempt to use the arms to support full body weight.
- DO NOT stand on the chair.

Fully Assembled Shower Chair

Cautions

Users with limited physical strength and/or mobility should be supervised and/or assisted while using the product. Prior to use, ensure the following;

1. There are no loose, broken or missing components and ensure all parts are secure.
2. When using the backrest, ensure the shower chair remains balanced. Excessive twisting or turning while seated may cause a loss of balance and/or damage to the product.
3. All legs are adjusted to the same preferred height and are touching the floor simultaneously at all times.
4. User evenly distributes weight between the arm supports when transferring in and out of the product.
5. The tips on the legs are not missing or contain any rips or cracks. Immediately replace missing or damaged tips.

Assembly and use

If exposed to cold temperatures, allow contents to come to room temperature before completing assembly. Remove contents from box, making sure you have all parts shown in (Figure 1):

- Seat (1)
- Armrest (2)

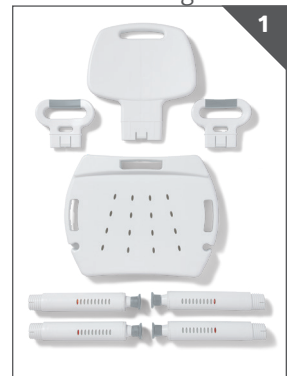
Figure 1

■ Height-adjustable Legs (4) ■ Backrest (1)

1. Place seat upside down and insert a leg assembly into any hole found at each corner. Turn counter-clockwise until leg is fully tightened. Repeat until all four legs have been secured to seat. Note: Legs will be flush with seat when fully-tightened. **(Figure 2)**
2. Insert bottom of each armrest into slots located on the short edges of the seat. **(Figure 3)**
3. Insert the bottom of the backrest into slot located at the rear of the seat. **(Figure 4)**
4. Adjust leg height to be the same on each of the four legs. Use the visible height indicator making sure it shows through the same window on each leg. Press down fully on center of seat to test stability before use.

Figure 2

Figure 3

Figure 4

Adjusting leg height

1. To decrease height, turn the foot piece on the leg, counter-clockwise. Repeat with all until they are the same preferred height.
2. To increase height, turn the foot piece on the leg clockwise. Repeat with all until they are the same preferred height.
3. Press down firmly on the center of the seat to test stability. Make any necessary adjustments to ensure the shower chair is leveled. **(Figure 5)**

Figure 5

Maintenance

Periodic visual inspection of this shower chair is recommended to ensure that all parts and hardware are secure, that components are in good working order and not worn, torn, frayed, or loose, and that there are no obstructions or impediments to normal, safe operation. If any part or mechanism appears to be unsafe or damaged, do not use the product. If you experience problems, please contact Medline at 1-800-MEDLINE | 1-800-268-2848.

Cleaning

1. Clean with regular household cleaning products. Use a non-abrasive cleaner with warm water. Wipe dry after cleaning.
2. DO NOT use abrasive powders, cleaners or scouring pads.

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Limited Warranty

Your Medline product is warranted to be free of defects in materials and workmanship for the duration of use by the original purchaser. This device was built to precise standards and carefully inspected prior to shipment. This limited warranty is an expression of our confidence in the materials and workmanship of our products and assures years of convenient, dependable service. In the event of a defect covered by this warranty, we will, at our option, repair or replace the device. This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters and grips, which are subject to normal wear and replacement.

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Some products may not be available for sale in Mexico or Canada. Please contact your Account Representative for additional information. We reserve the right to correct any errors that may occur within this manual. ©2021 Medline Industries, LP. Guardian and Medline are registered trademarks of Medline Industries, LP.

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Medline Industries, LP, Three Lakes Drive, Northfield, IL 60093 USA.

Medline United States | États Unis | Estados Unidos

1-800-MEDLINE (633-5463)

medline.com | info@medline.com

Medline Canada

1-800-268-2848 | medline.ca | canada@medline.com

Medline México

01-800-831-0898 | medlinemexico.com | mexico@medline.com

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