# GUARDIAN<sup>®</sup>

User's Guide

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## General Warnings

To ensure the safe assembly and operation of the product, these instructions must be followed. FAILURE TO FOLLOW INSTRUCTIONS MAY RESULT IN SERIOUS INJURY TO THE USER OR DAMAGE TO THE PRODUCT. If you are unable to understand these warnings, cautions and/or instructions, contact a healthcare professional, dealer or Medline before attempting to use this product.

- DO NOT assemble and use this product without first reading and understanding this user guide, otherwise injury or damage may occur.
- DO NOT use if components are missing or are damaged, contact Medline at 1-800-MEDLINE | 1-800-268-2848.
- DO NOT exceed weight capacity of 250 lbs (113 kg).
- DO NOT make any adjustments while using the product.
- DO NOT stand on the shower stool.

## Cautions

Users with limited physical strength and/or mobility should be supervised and/or assisted while using the product. Prior to use, ensure the following:

1. There are no loose, broken or missing components and ensure all parts are secure.
2. All legs are adjusted at the same preferred height and are touching the floor simultaneously at all times.
3. All screws, nuts and/or bolts are tightened.
4. All push buttons protrude fully and legs are securely locked in place.

## Assembly and use

Frame is shipped pre-assembled. If exposed to cold temperatures, allow contents to come to room temperature before completing assembly. Remove contents from box, making sure you have all parts described here.

- seat
- leg assembly
- legs

### (Figure 1)

1. Open leg assembly and press cross bars firmly into seat clip. **(Figure 2)**
2. Insert the leg extensions imprinted with *S1* into the leg assemblies imprinted with *S1*. **(Figure 3)**
3. Insert the leg extensions imprinted with *S2* into the leg assemblies imprinted with *S2*. **(Figure 4)**

NOTE: Ensure to adjust ALL the legs to the same preferred height.

4. To fold for storage, unclip one side of seat and fold legs flat.

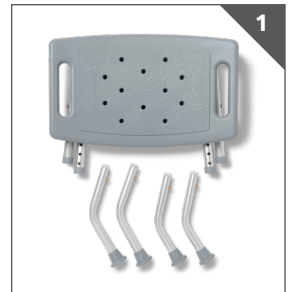
Figure 1

Figure 2

## Adjusting leg height

Before using the product, make sure all legs are set to the same preferred height.

1. Depress the push button and slide each leg to the desired height. **(Figure 5)**
2. Align the push button to the proper hole and let the push button snap into place.
3. Repeat until all legs are adjusted to the same preferred height.
4. Ensure legs are locked into place by pushing down and pulling up on each leg.

## Maintenance

Periodic visual inspection of this shower stool is recommended to ensure that all parts and hardware are secure, that components are in good working order and not worn, torn, frayed, or loose, and that there are no obstructions or impediments to normal, safe operation. If any part or mechanism appears to be unsafe or damaged, do not use the product. If you experience problems, please contact Medline at 1-800-MEDLINE | 1-800-268-2848.

## Cleaning

1. Clean with regular household cleaning products. Use a non-abrasive cleaner with warm water. Wipe dry after cleaning.
2. DO NOT use abrasive powders, cleaners or scouring pads.

**Figure 3**

**Figure 4**

**Figure 5**

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## Limited Warranty

Your Medline product is warranted to be free of defects in materials and workmanship for the duration of use by the original purchaser. This device was built to precise standards and carefully inspected prior to shipment. This limited warranty is an expression of our confidence in the materials and workmanship of our products and assures years of convenient, dependable service. In the event of a defect covered by this warranty, we will, at our option, repair or replace the device. This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters and grips, which are subject to normal wear and replacement.

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Some products may not be available for sale in Mexico or Canada. Please contact your Account Representative for additional information. We reserve the right to correct any errors that may occur within this manual. ©2021 Medline Industries, LP. Guardian and Medline are registered trademarks of Medline Industries, LP.

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Medline Industries, LP, Three Lakes Drive, Northfield, IL 60093 USA.

Medline United States  
1-800-MEDLINE (633-5463)  
medline.com | info@medline.com

Medline Canada

1-800-268-2848 | medline.ca | canada@medline.com

Medline México

01-800-831-0898 | medlinemexico.com | mexico@medline.com

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