

## Elastic Tubular Bandage Compression Sizing Chart

Provides sustained pressure and support for: strains, sprains and soft tissue injuries. Support following cast removal, general edema, post-graft pressure for burn patients and rib cage injuries.



### Instructions for Use

1. Cut the length of the elastic tubular bandage to fit the limb or area to be covered - allow an extra 1 in for overlap
2. Draw the bandage carefully over the limb or area like a stocking
3. A single layer only is recommended

### SIZING CHART

MFR #	SIZE	SPECS	LOW PRESSURE	MEDIUM PRESSURE	HIGH PRESSURE
			CIRCUMFERENCE 5 - 10 mmHG	CIRCUMFERENCE 10 - 20 mmHG	CIRCUMFERENCE 20 - 30 mmHG
182-13112C	Size C	2.75 in x 11 yds	9.4 - 10.6 in	10.6 - 11.8 in	11.8 - 13.8 in
182-13113D	Size D	3 in x 11 yds	10.6 - 11.8 in	11.8 - 13.8 in	13.8 - 17.3 in
182-13114E	Size E	3.5 in x 11 yds	11.8 - 13.8 in	13.8 - 17.3 in	17.3 - 18.9 in
182-13115F	Size F	4 in x 11 yds	13.8 in - 17.3 in	17.3 - 22.8 in	22.8 - 25.2 in
182-13116G	Size G	4.5 in x 11 yds	17.3 - 22.8 in	22.8 - 25.2 in	25.2 - 28.3 in
182-13117J	Size J	6.75 in x 11 yds	25.2 - 28.3 in	28.3 - 33.8	33.8 - 37.8 in