# **GUARDIAN®**

Family-owned. Trusted by hospitals for over 50 years.

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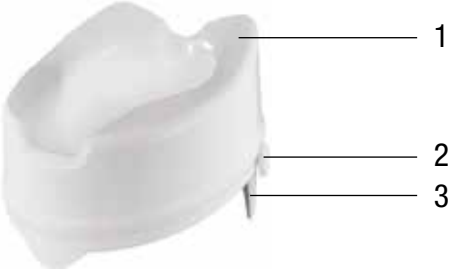
**LIFETIME  
LIMITED  
WARRANTY**

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**REF G4-400RX1**

**Note:** Check all parts for shipping damage. In case of shipping damage, **DO NOT** use. Contact Medline at 1-800-MEDLINE.

NO.	DESCRIPTION	QTY
1	TOILET SEAT	1
2	ADJUSTMENT KNOBS	2
3	L CLIP	2

## INSTALLATION

The raised toilet seat has been designed to accommodate a wide variety of toilet bowl types. However, certain bowl types will require the removal of the existing seat and/or lid to ensure proper fit when installed.

1. Lift the seat and lid of the toilet.
2. Rotate the adjustment knobs counter clockwise until they are loose enough to fit onto your toilet bowl.
3. Place the raised toilet seat onto the bowl. Center the raised toilet seat and push the toilet seat back on the bowl. Ensure your toilet seat is pushed all the way back before proceeding to Step 4.
4. Rotate both adjustment knobs clockwise until the raised toilet seat is snug on the bowl. Hand tighten only.

## CAUTIONS

» The raised toilet seat **MUST** be leveled and **MUST** be secured to the toilet before use.

- » DO NOT use if the toilet seat does not support your body weight. Weight capacity of this item is 250 lbs(113 kg).
- » When the raised toilet seat is properly installed, the unit will sit flat on the top of the lip of the bowl and MUST fit securely against the inner rim of the bowl.

## WARNINGS

- » **SAFETY SUMMARY:** The following recommendations are made for the safe assembly of the raised toilet seat.
- » DO NOT assemble or use this equipment without first reading and understanding this instruction sheet. If you are unable to understand the Warnings, Cautions or Instructions, contact a healthcare professional, dealer or technical personnel before attempting to install this equipment. Otherwise, injury or damage may occur.
- » Certain toilet fixtures will prevent the locking raised toilet seat from working properly and/or will damage the locking raised toilet seat. If the toilet bowl has on its top or inner ring any raised or protruding points or sections, then the locking raised toilet seat should not be used.
- » BEFORE EACH USE, ensure that ALL attaching hardware is securely tightened.
- » Center your weight on the raised toilet seat. The seat may tip if you lean too far to any side.
- » Users with limited physical strength and/or mobility should be supervised and/or assisted when using the raised toilet seat.
- » Rotate the adjustment knob by hand only to prevent over-torque of the assembly. Use of a wrench or other tool will break the screw assembly.
- » DO NOT FORCE knob as this may damage the knob and/or the raised toilet seat.
- » The raised toilet seat is not designed for permanent installation. The unit must be inspected prior to each use to ensure a proper fit.
- » If, for any reason, a proper fit in installation cannot be achieved, **DO NOT** use and please contact Medline at 1-800-MEDLINE.

## CLEANING INSTRUCTIONS

1. Use a non-abrasive detergent or household cleaner with warm water.
2. Rinse with warm water and wipe dry.
3. DO NOT use an abrasive cleaner or cloth with this product as damage may occur.

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## LIMITED WARRANTY

Your Medline product is warranted to be free of defects in materials and workmanship for the duration of use by the original purchaser. This device was built to precise standards and carefully inspected prior to shipment. This limited warranty is an expression of our confidence in the materials and workmanship of our products and assures years of convenient, dependable service. In the event of a defect covered by this warranty, we will, at our option, repair or replace the device. This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters and grips, which are subject to normal wear and replacement.

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---

**GUARDIAN**<sup>®</sup>

Medline Industries, LP,  
Three Lakes Drive,  
Northfield, IL 60093 USA

---

Medline United States  
1-800-MEDLINE (633-5463)  
www.medline.com  
www.info@medline.com

Medline Canada  
1-800-268-2848  
www.medline.com  
Canada@medline.com

Medline Mexico  
01-800-831-0898  
medlinemexico.com  
mexico@medline.com

Some products may not be available for sale in Mexico or Canada. Please contact your Account Representative for additional information. We reserve the right to correct any errors that may occur within this manual. ©2022 Medline Industries, LP. Guardian and Medline are registered trademarks of Medline Industries, LP.

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