**User Instructions and Warranty**

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## General Warnings

To ensure the safe assembly and operation of this product, these instructions must be followed. **FAILURE TO FOLLOW INSTRUCTIONS MAY RESULT IN SERIOUS INJURY TO THE USER OR DAMAGE TO THE PRODUCT.** If you are unable to understand these warnings, cautions and/or instructions, contact a healthcare professional, dealer or Medline before attempting to use this product.

- ⚠ **DO NOT** assemble and use this product without first reading and understanding this user's guide, otherwise injury or damage may occur.
- ⚠ **DO NOT** use if components are missing or are damaged. Contact Medline at **1-800-MEDLINE (633-5463) | 1-800-268-2848**.
- ⚠ **DO NOT** exceed weight capacity of **300 lb (136 kg)**.
- ⚠ **DO NOT** make any adjustments while using the product.
- ⚠ **DO NOT** stand on the product.
- ⚠ **DO NOT** excessively twist or turn while seated on the bench as this may cause a loss of balance and/or damage to the product.
- ⚠ **DO NOT** attempt to use the arms to support full body weight.
- ⚠ **DO NOT** attempt to use the backrest to support full body weight.
- ⚠ **DO NOT** use product for anything other than its intended use.
- ⚠ **DO NOT** use product if wobbly, unstable, or not level.
- ⚠ **DO NOT** use product on uneven surfaces.
- ⚠ **DO NOT** "plop" or suddenly drop body weight onto the transfer bench/commode seat. Ease gently onto the seat when attempting to sit. Use caution when attempting to sit as the seat may be slippery.
- ⚠ **WARNING:** Transfer bench can be very slippery when wet.

## Cautions

Users with limited physical strength and/or mobility should be supervised and/or assisted while using the product. Prior to use, ensure the following:

1. The bench is completely unfolded. Failure to do so may result in possible injury or damage.
2. All push buttons are fully protruded and are properly engaged.
3. There are no loose, broken or missing components.
4. All parts are secure.
5. The bench must be balanced at all times.
6. All four legs touch the floor simultaneously at all times.
7. The tips are not worn, broken or missing. Replace any missing or damaged tips before use.
8. All legs are at the same height adjustment level.
9. Ensure seatbelt is fastened properly before use.

# Assembly

If exposed to cold temperatures, allow contents to come to room temperature before completing assembly. Remove contents from box. Ensuring all parts listed are included.

- Backrest (1 pc)
- Armrest (1 pc)
  - Knobs (2 pcs)
- Main Frame with Seat (1 pc)
- Leg Tubes (4 pcs)
  - Suction Tip (2 pcs)
  - Regular Tip (2 pcs)
- Soap Dish (1 pc)
- Seat Belt (1 pc)

Note: Seatbelt will come attached.

1. Place transfer bench seat upside down on a flat surface (Figure 1).
2. Unfold each leg then pull up on black bars to lock (Figure 1).
3. Attach two legs with the same tip to one side by depressing the pushbuttons and inserting into the receptacle with the matching "A" until the pushbutton snaps and locks into place. Repeat for second leg (Figure 2).
4. Attach the other two legs with the same tips to the other side by depressing pushbutton and inserting into the receptacle with matching "B" until the pushbutton snaps and locks into place. Repeat for second leg.
5. Turn transfer bench right-side up and ensure all leg tips are at the same height.
6. Insert backrest into the slots in the back of the seat (Figure 3).
7. Attach handle on desired side of seat by placing into the holes on the seat and insert the thumb screw to secure (Figure 4 & 5).
8. Attach soap on desired side by placing into the front slot (Figure 6).

# Use

## Wipe bath seat dry after each use.

**Caution:** Evenly distribute weight between the arm supports when transferring in and out of the bench.

## Instructions for use

To operate the sliding mechanism, pull up on the black handle under the seat and slide to the desired direction until seat clicks into place.

**NOTE:** There are two positions where the seat locks into place, one to the right and one to the left.

## Folding

1. On one end pull down on the black bars to unlock that side of legs.
2. Pull both red locking levers away from frame simultaneously until both locking pins disengage.

**NOTE:** Legs can stay on when folding or come off.

3. Repeat steps 1-2 for the other side of the bench.

## Leg height adjustment

1. Before use, adjust the height of each leg to your specific needs, making sure all legs are at the same height. When desired height is reached, lock legs into place by aligning buttons into proper holes.
2. Ensure that the legs are locked into place by pushing down and pulling up on each leg.

## Maintenance

Periodic visual inspection of this product is recommended to ensure that all parts and hardware are secure, that components are in good working order and not cracked, broken or fractured, and that there are no obstructions or impediments to normal, safe operation. If any part or mechanism appears to be unsafe or damaged, do not use the product. If you experience problems, please contact Medline at **1-800-MEDLINE (633-5463) | 1-800-268-2848**.

## Cleaning

1. Clean with a non-abrasive cleaner or household cleaner with warm water.
2. Rinse with warm water and wipe dry after cleaning.
3. **DO NOT** use abrasive powders, cleaners, cloths, or scouring pads.

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## Limited Warranty

Your Medline product is warranted to be free of defects in materials and workmanship for the duration of use by the original purchaser. This device was built to precise standards and carefully inspected prior to shipment. This limited warranty is an expression of our confidence in the materials and workmanship of our products. In the event of a defect covered by this warranty, we will, at our option, repair or replace the device. This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as tip accessories, casters and grips, which are subject to normal wear and replacement.

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Some products may not be available for sale in Mexico or Canada. Please contact your Account Representative for additional information. We reserve the right to correct any errors that may occur within this manual.

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