



QWICK™

SUPER ABSORBENT WICKING FIBER




See application video at www.medline.com/awcvideos

PREPARATION:

Step 1: Cleanse wound environment using an appropriate wound cleanser.

Step 2: Ensure skin surrounding wound is dry and apply appropriate skin prep, as preferred.


APPLICATION:

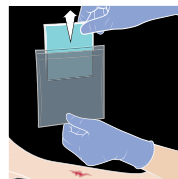

1. Remove dressing from package. Qwick™ may be cut to size.



2. Place dressing on wound with blue side facing away from wound.

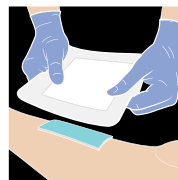

3. Secure in place with appropriate cover dressing, such as bordered or rolled gauze.

© 2018 Medline Industries, Inc. Medline is a registered trademark of Medline Industries, Inc. MKT1895699 / e18050

SUPER ABSORBENT WICKING FIBER