



- When used as a protectant against body fluids, feces, or urine etc., and no adhesive products are applied to skin:
  - Reapplication of Sureprep Rapid Dry Barrier Film is recommended every 24-72 hours, depending on frequency of cleaning.
  - In extreme cases (e.g. constant diarrheal stooling) with very frequent cleansing, more frequent applications may be necessary (i.e. every 12-24 hours).
- If desired, the film can be removed by using most medical adhesive removers as directed. Clean and dry the involved area and reapply Sureprep Rapid Dry Barrier Film.

#### INGREDIENTS

Hexamethyldisiloxane, Acrylate Copolymer.

#### STORAGE INFORMATION

Store away from direct sunlight at ambient temperature and humidity.

#### HOW SUPPLIED

MSC1605	1 mL wipe	25/box	4 bx/cs
MSC1610	1 mL wand	25/box	4 bx/cs
MSC1613	3 mL wand	25/box	4 bx/cs

#### [www.medline.com](http://www.medline.com)

Medline and Sureprep are registered trademarks of Medline Industries, LP. Manufactured in Ireland for Medline Industries, LP, Three Lakes Drive, Northfield, IL 60093 USA. **1-800-MEDLINE** V3 RC23FNS

# SUREPREP® RAPID DRY

## NO-STING BARRIER FILM

### Instructions for Use

**REF** MSC1605

**REF** MSC1610

**REF** MSC1613

## **PRODUCT DESCRIPTION**

Sureprep Rapid Dry Barrier Film is a polymeric solution which forms a uniform film when applied to the skin. This product is dispersed in a unique, non-stinging, non-cytotoxic solvent which dries rapidly. The film is colorless, transparent, and possesses good oxygen and moisture vapor permeability. Sureprep Rapid Dry Barrier Film helps to protect intact skin from irritation caused by body fluids such as urine and feces, digestive fluids, wound drainage, and friction or adhesives.

## **INDICATIONS FOR USE**

Sureprep Rapid Dry Barrier Film is a liquid intended for use as a film-forming product, that upon application to intact skin forms a long lasting waterproof barrier, which acts as a protective interface between the skin and bodily wastes, fluids, adhesive products, and friction. It is intended as a primary barrier against irritation from body fluids.

## **CONTRAINDICATIONS**

Sureprep Rapid Dry Barrier Film is not to be used as the only covering in situations that require dressing protection from bacterial contamination/ penetration, e.g. I.V. therapy catheter sites, full or partial thickness wounds, or infected skin.

## **WARNINGS**

Sureprep Rapid Dry Barrier Film, in liquid form (prior to drying completely), is flammable; use in a well ventilated area. Avoid using around flames and sources of ignition. Keep out of reach of children. Do not use if pack is damaged or open.

## **PRECAUTIONS**

1. Should redness or any other signs of irritation appear, or if redness or signs of irritation persist, cease the use of Sureprep Rapid Dry Barrier Film and consult a physician.
2. Use of other barrier products, ointments, creams or lotions may significantly reduce the effectiveness of Sureprep Rapid Dry Barrier Film.

## **DIRECTIONS FOR USE**

1. Skin should be clean and dry prior to application of Sureprep Rapid Dry Barrier Film.
2. Remove wipe or applicator from sachet and apply a uniform coating of film over entire area of concern.
3. If an area is missed, reapply to that area only after first application of Sureprep Rapid Dry Barrier Film has dried (approximately 30 seconds).
4. If Sureprep Rapid Dry Barrier Film is applied to area with skin folds or other skin-to-skin contact, make sure that skin contact areas are separated to allow the film to thoroughly dry before returning to normal position.
5. When used under adhesive tapes, dressings, or devices:
  - (a) Allow Sureprep Rapid Dry Barrier Film to thoroughly dry before covering with dressing or adhesive products.
  - (b) Reapplication of Sureprep Rapid Dry Barrier Film is necessary each time dressing and/or adhesive products are changed as the barrier film is removed by the adhesive.