


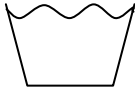


# SURGICAL- Laundering Recommendations

Reusable Gowns, Drapes, and Wrappers:  
AAMI Level 3 and 4 Xalt, Blockade, Stericloth, Resistat, Rip-Stop, Angelstat


 <b>WASHING</b>	Operation	Temperature	Time
	Suds	72°C/162°F	40 min
	Rinse	60°C/140°F	5 min
	Rinse	60°C/140°F	10 min
	Wet Tumble	----	5 min

**DO NOT** use Softener

*\*For best results use batch washer*

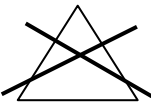  
**NO BLEACH**


**DO NOT** use bleach - will damage barrier properties

  
**DRYING**

Tumble Dry - 66°C +/-5°C | 150°F +/-10°F  
**MAX** temperature 71°C/160°F  
With final 10 minute cool-down until fabric is dry

  
**NO IRONING**

**DO NOT** flat-work iron any surgical product

  
**PREVAC  
STEAM STERILIZATION**

Product provided non-sterile, to sterilize product:  
134°C/274°F  
4 minute exposure  
20 minute exhaust

**All Products** Must be laundered prior to use  
Only wash with like items  
Only wash with like colors

**AAMI Life Cycle** Products will pass labeled AAMI Level for up to 75 processings when following above recommendations

**DO NOT** Use Fabric Softener. Fabric softener inhibits absorbency / wicking  
**DO NOT** Overdry. Overdrying can lead to melting / hardening, sheen, static and permanent creasing

\*Surgical materials meet the Standard for Flammability of Clothing Textiles, CPSC 16 CFR Part 1610, Class 1 (formerly CS 191-53).