Transfer Bench with Back and Arm

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Weight Capacity

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350 lb (158 kg)

NOTE: Before assembly, ensure that all parts and quantities match the parts list. Inspect all components for shipping damage. If any parts are missing or damaged, contact your dealer/retailer for replacements.

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

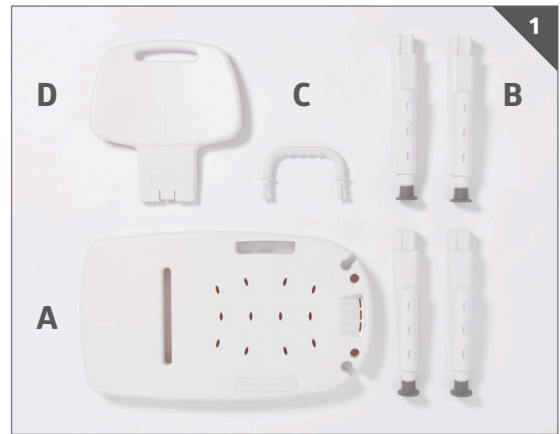
G3-500KRX1 Parts List:

- (A) Seat -1 each
 - (B) Leg - 4 each
 - (C) Arm -1 each
 - (D) Back -1 each
 - (E) Plastic insert for seat -1 each (not shown)
- (Figure 1)

To Assemble:

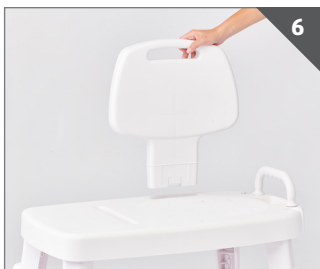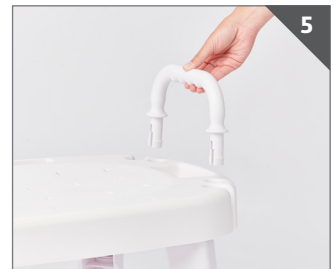1. Place seat (A) upside down on a flat surface (Figure 2). Insert leg (B) into the corner recess of the seat by applying a firm downward pressure until the leg is fully engaged (Figure 3). Repeat this step with the remaining 3 legs (Figure 4).
2. Place the bench upright and insert the arm (C) into the holes on the side of the bench until fully engaged (Figure 5).
3. Insert back (D), after determining which way the bench will be used, into the slot at the rear of the seat until fully engaged (Figure 6).
4. Insert the plastic insert (E) (not pictured) into the slot opposite the backrest.
5. To adjust the legs for proper height or uneven surfaces, rotate the leg extension and disengage from the leg (Figure 7). Insert the leg extension into the desired slot until adjustment is fully engaged (Figure 8). If having difficulty adjusting the leg, remove the tip from the leg and, using a screwdriver, gently twist the leg until it is disengaged (Figure 9). Place the leg extension into the proper placement and insert into slot until fully engaged. Replace the tip before using the bench.

To Disassemble:

The back has a lock tab that, when depressed, allows the back to be pulled out of the slots (Figure 10). To remove the arms, pinch together the tabs on the arms (when assembled, tabs will be under the seat). Do not remove legs as this may cause damage to the product. The legs were not designed to be removed after assembly.

Care and Maintenance:

Periodic visual inspection of this product is recommended to ensure that all parts and hardware are secure, that components are in good working order and not cracked, broken or fractured, and that there are no obstructions or impediments to normal, safe operation. If any part or mechanism appears to be unsafe or damaged, do not use the product. If you experience problems, please contact Medline at 1-800-MEDLINE (633-5463) | 1-800-268-2848.

Cleaning Instructions:

1. Clean with a non-abrasive cleaner or household cleaner with warm water.
2. Rinse with warm water and wipe dry after cleaning.
3. **DO NOT** use abrasive powders, cleaners, cloths, or scouring pads.

General Warnings and Installation Instructions:

To ensure the safe assembly and operation of this product, these instructions must be followed. **FAILURE TO FOLLOW INSTRUCTIONS MAY RESULT IN SERIOUS INJURY TO THE USER OR DAMAGE TO THE PRODUCT.** If you are unable to understand these warnings, cautions and/or instructions, contact a healthcare professional, dealer or Medline before attempting to use this product.

General Warnings:

DO NOT assemble and use this product without first reading and understanding this instruction sheet. If you are unable to understand these warnings, cautions, and instructions, contact a healthcare professional or dealer before attempting to install this equipment. Failure to follow instructions may result in serious injury.

- **DO NOT** place transfer bench on uneven surfaces.
- **DO NOT** use if wobbly, unstable or not level.
- **DO NOT** exceed user weight capacity of 350 lb (158 kg).
- **DO NOT** twist or turn while seated on transfer bench, this may cause frame to bend.
- **DO NOT** stand on transfer bench.
- **DO NOT** use transfer bench for any purpose that it is not intended to be used for.
- **DO NOT** use if components are missing or are damaged. Contact Medline at **1-800-MEDLINE**.
- **DO NOT** make any adjustments while using the product.
- **DO NOT** attempt to use the armrest to support full body weight.
- **DO NOT** attempt to use the backrest to support full body weight.

Cautions:

- Prior to each use check that all joints and fittings on the transfer bench are tight.
- Make sure all four legs are adjusted evenly and are free of any obstructions prior to use.
- Check rubber tips for rips, cracks or wear. If these conditions exist, **DO NOT** use the tips and replace **IMMEDIATELY**. (Rubber Tips can be ordered—item MDS89755T)
- Users with limited physical strength and/or mobility should be supervised and/or assisted while using the product. Prior to use, ensure the following:
 - All push buttons are fully protruded and are properly engaged.
 - There are no loose, broken or missing components.
 - Ensure the chair remains balanced.
 - Excessive twisting or turning while seated on the chair may cause a loss of balance and/or damage to the product.
 - All four legs touch the floor simultaneously at all times.

Instructions for Use:

Leg Height Adjustment:

1. Before use, adjust the height of each leg to your specific needs, making sure all legs are at the same height. When desired height is reached, lock legs into place by aligning buttons into proper holes (Figure 7-9).
2. Ensure that the legs are locked into place by pushing down and pulling up on each leg.

Keep Your Receipt!

Contact us at 1-800-MEDLINE, Monday-Friday 8AM-5PM CST if you encounter any problems or for information regarding repairs or maintenance. **Medline disclaims all other warranties including but not limited to the implied warranties of merchantability and fitness for a particular purpose.**

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Limited Warranty

Your Medline product is warranted to be free of defects in materials and workmanship for the duration of use by the original purchaser. This device was built to precise standards and carefully inspected prior to shipment. This limited warranty is an expression of our confidence in the materials and workmanship of our products and assures years of convenient, dependable service. In the event of a defect covered by this warranty, we will, at our option, repair or replace the device. This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as tip accessories, casters and grips, which are subject to normal wear and replacement.

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Some products may not be available for sale in Mexico or Canada. Please contact your Account Representative for additional information. We reserve the right to correct any errors that may occur within this manual. ©2023 Medline Industries, LP. Medline is a registered trademark of Medline Industries, LP.

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