# User Instructions and Warranty

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## General Warnings

To ensure the safe assembly and operation of this product, these instructions must be followed. **FAILURE TO FOLLOW INSTRUCTIONS MAY RESULT IN SERIOUS INJURY TO THE USER OR DAMAGE TO THE PRODUCT.** If you are unable to understand these warnings, cautions and/or instructions, contact a healthcare professional, dealer or Medline before attempting to use this product.

- ⚠ **DO NOT** assemble and use this product without first reading and understanding this user's guide, otherwise injury or damage may occur.
- ⚠ **DO NOT** use if components are missing or are damaged. Contact Medline at **1-800-MEDLINE (633-5463) | 1-800-268-2848**.
- ⚠ **DO NOT** exceed weight capacity listed on the product.
- ⚠ **DO NOT** make any adjustments while using the product.
- ⚠ **DO NOT** stand on the product.
- ⚠ **DO NOT** excessively twist or turn while seated on the bench as this may cause a loss of balance and/or damage to the product.
- ⚠ **DO NOT** attempt to use the arms to support full body weight.
- ⚠ **DO NOT** attempt to use the backrest to support full body weight.
- ⚠ **DO NOT** use product for anything other than its intended use.
- ⚠ **DO NOT** use product if wobbly, unstable, or not level.
- ⚠ **DO NOT** use product on uneven surfaces.
- ⚠ **DO NOT** "plop" or suddenly drop body weight onto the transfer bench/commode seat. Ease gently onto the seat when attempting to sit.
- ⚠ **WARNING:** Transfer bench can be slippery when wet. Use caution when attempting to sit or transfer.
- ⚠ **WARNING:** Ensure weight is evenly distributed between the arm supports, if applicable, when transferring in and out of the bench.

## Cautions

Users with limited physical strength and/or mobility should be supervised and/or assisted while using the product. Prior to use, ensure the following:

1. The bench is completely unfolded. Failure to do so may result in possible injury or damage.
2. All push buttons are fully protruded and are properly engaged.
3. There are no loose, broken or missing components.
4. All parts are secure.
5. The bench must be balanced at all times.
6. Excessive twisting or turning while seated on the bench may cause a loss of balance and/or damage to the product.
7. All four legs touch the floor simultaneously at all times.
8. User evenly distributes weight between the arm supports when transferring in and out of the bench.
9. The tips are not worn, broken or missing. Replace any missing or damaged tips before use.
10. Ensure all legs are at the same height adjustment level before use.

# Assembly

Ensure that you have all of the necessary components before assembling your bath bench. If any parts are missing, **DO NOT USE**, please contact Medline at **1-800-MEDLINE (633-5463) | 1-800-268-2848**. If exposed to cold temperatures, allow contents to come to room temperature before completing assembly.

- 1 Bench—with preassembled main frame
- 1 Backrest—with preassembled backrest tubes
- 2 Backrest Support Tubes
- 1 Armrest
- 2 Suction Cup Leg Tubes
- 2 Regular Tip Leg Tubes
- 1 Padded Bench—with preassembled main frame
- 1 Backrest—with preassembled backrest tubes
- 2 Backrest Support Tubes
- 1 Armrest
- 2 Suction Cup Leg Tubes
- 2 Regular Tip Leg Tubes
- 1 Bench—with preassembled main frame
- 1 Backrest—with preassembled backrest tubes
- 2 Backrest Support Tubes
- 1 Armrest
- 2 Suction Cup Leg Tubes
- 2 Regular Tip Leg Tubes

G3-100KBX1, G3-100KBX2

G3-100KBPX1, G3-100KBPX2

G3-200KBX1, G3-200KBX2

While the components of the three product models vary, their installation procedures remain consistent which same as G3-100KBX1.

## Use

1. Remove contents from box.
2. Lay transfer bench upside down on a flat surface.
3. Attach leg with the large suction cup (marked "A") by depressing the pushbutton and inserting into the receptacle tube marked "A" until the pushbutton snaps and locks into place. Repeat for second leg (**Figure 1**).
4. Attach leg with small rubber tip (marked "B") by depressing the pushbutton and inserting into the receptacle tube marked "B" until the pushbutton snaps and locks into place. Repeat for second leg (**Figure 2**).
5. Turn transfer bench right-side-up. Depress armrest pushbuttons and slide into the armrest receptacle tubes with armrest arching outwards. (**Figure 3**).

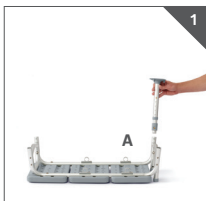
Figure 1

Figure 2

Figure 3

6. To assemble backrest, insert pole into backrest support by depressing pushbutton and sliding into receptacle tube until it locks into place. Repeat for second pole (**Figure 4**).

7. Insert backrest into base by depressing pushbuttons and sliding supports through both receptacle tubes.

**CAUTION:** Backrest must be locked and secured in each receptacle tube. (**Figure 5**).

**NOTE:** Back support is reversible to accommodate left or right tub entry. You can also change the seat depth of your transfer bench by pushing the pins under the seat and sliding the tubes forward until the pins lock into place.

8. Place legs with suction cups into bath tub.

**Figure 4**

**Figure 5**

## Leg Height Adjustment

1. Before use, adjust the height of each leg to your specific needs, making sure all legs are at the same height. When desired height is reached, lock legs into place by aligning buttons into proper holes.
2. Ensure that the legs are locked into place by pushing down and pulling up on each leg.

## Maintenance

Periodic visual inspection of this product is recommended to ensure that all parts and hardware are secure, that components are in good working order and not cracked, broken or fractured, and that there are no obstructions or impediments to normal operation.

Wipe bath seat dry after each use. Drain any water that has accumulated in the seat or backrest. If any part or mechanism appears to be unsafe or damaged, **DO NOT** use the product.

If you experience problems, please contact Medline at

**1-800-MEDLINE (633-5463) | 1-800-268-2848.**

## Cleaning

1. Clean with a non-abrasive cleaner or household cleaner with warm water.
2. Rinse with warm water and wipe dry after cleaning.
3. **DO NOT** use abrasive powders, cleaners, cloths, or scouring pads.

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## Limited Warranty

Your Medline product is warranted to be free of defects in materials and workmanship for the duration of use by the original purchaser. This limited warranty is an expression of our confidence in the materials and workmanship of our products. In the event of a defect covered by this warranty, we will, at our option, repair or replace the device. This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as tip accessories, casters and grips, which are subject to normal wear and replacement.

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Some products may not be available for sale in Mexico or Canada. Please contact your Account Representative for additional information. We reserve the right to correct any errors that may occur within this manual. ©2025 Medline Industries, LP. Medline is a registered trademark of Medline Industries, LP.

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