

GUARDIAN®

Family-owned. Trusted by hospitals for over 50 years.



BATHTUB TRANSFER BENCH

USER INSTRUCTIONS AND WARRANTY

G3-100KBX1, G3-100KBX2, G3-100KBPX1, G3-100KBPX2: Weight Capacity 400 lbs
G3-200KBX1, G3-200KBX2: Weight Capacity 500 lbs



This product is not made with dry natural rubber.

REF	G3-100KBX1	G3-100KBX2
	G3-100KBPX1	G3-100KBPX2
	G3-200KBX1	G3-200KBX2

LIFETIME
LIMITED
WARRANTY

MAINTENANCE

Periodic visual inspection of this transfer bench is recommended to ensure that all parts and hardware are secure, that components are in good working order and not worn, torn, frayed, or loose, and that there are no obstructions or impediments to normal, safe operation. If any part or mechanism appears to be unsafe or damaged, do not use the product. If you experience problems, please contact Medline at 1-800-MEDLINE

ASSEMBLY:

1. Remove contents from box.
2. Lay transfer bench upside down on a flat surface.
3. Attach leg with the large suction cup (marked “A”) by depressing the pushbutton and inserting into the receptacle tube marked “A” until the pushbutton snaps and locks into place. Repeat for second leg (Figure 1).



Figure 1

4. Attach leg with small rubber tip (marked “B”) by depressing the pushbutton and inserting into the receptacle tube marked “B” and locks into place. Repeat for second leg (Figure 2).

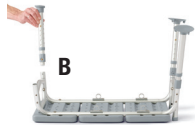


Figure 2

5. Turn transfer bench right-side-up. Depress armrest pushbuttons and slide into the armrest receptacle tubes with armrest arching outwards. (Figure 3).



Figure 3

6. To assemble backrest, insert pole into backrest support by depressing pushbutton and sliding into receptacle tube until it locks into place. Repeat for second pole (Figure 4).



Figure 4



Figure 5

7. Insert backrest into base by depressing push pins Insert backrest into base by depressing pushbuttons and sliding supports through both receptacle tubes.

CAUTION: Backrest must be locked and secured in each receptacle tube. (Figure 5).

NOTE: Back support is reversible to accommodate left or right tub entry. You can also change the seat depth of your transfer bench by pushing the pins under the seat and sliding the tubes forward until the pins lock into place.

8. Place legs with suction cups into bath tub.

Leg Height Adjustment

1. Before use, adjust the height of each leg to your specific needs, making sure all legs are at the same height. When your desired height is reached, lock legs into place by aligning buttons with holes until they “snap” into place.
2. Make sure legs are locked into place by pushing down or by pulling up on legs.

CAUTIONS:

1. If there is shipping damage - DO NOT USE. Contact dealer/carrier for further instructions.
2. Make certain that the height adjusting snap buttons fully protrude through the same respective hole of each leg extension. This will ensure that the leg extensions are securely locked in position and an even height is achieved.
3. Ensure ALL screws, nuts and/or bolts are tightened.
4. Check the rubber tips on the leg extensions for rips, wear or if they are missing. Immediately replace any or all if any of these imperfections exist.
5. All four leg extensions with rubber tips MUST touch the floor simultaneously at all times.
6. DO NOT stand on chair.
NOTE: To maintain the beauty of the original finish, clean with mild soap and rinse with warm water. Wipe bath seat dry after each use.

LIMITED WARRANTY

Your Medline product is warranted to be free of defects in materials and workmanship for the duration of use by the original purchaser. This device was built to precise standards and carefully inspected prior to shipment. This limited warranty is an expression of our confidence in the materials and workmanship of our products and assures years of convenient, dependable service. In the event of a defect covered by this warranty, we will, at our option, repair or replace the device. This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters and grips, which are subject to normal wear and replacement.

GUARDIAN®



Medline Industries, Inc.
Three Lakes Drive
Northfield, IL 60093

Medline United States
1-800-MEDLINE (633-5463)
www.medline.com
www.info@medline.com

Medline Canada
1-800-396-6996
www.medline.ca
Canada@medline.com

Medline Mexico
01-800-831-0898
medlinemexico.com
mexico@medline.com

Some products may not be available for sale in Mexico or Canada. Please contact your Account Representative for additional information. We reserve the right to correct any errors that may occur within this manual. ©2019 Medline Industries, Inc. Guardian and Medline are registered trademarks of Medline Industries, Inc.