Medline Transport Chair  
Fauteuil de transport Medline  
Silla de transporte Medline

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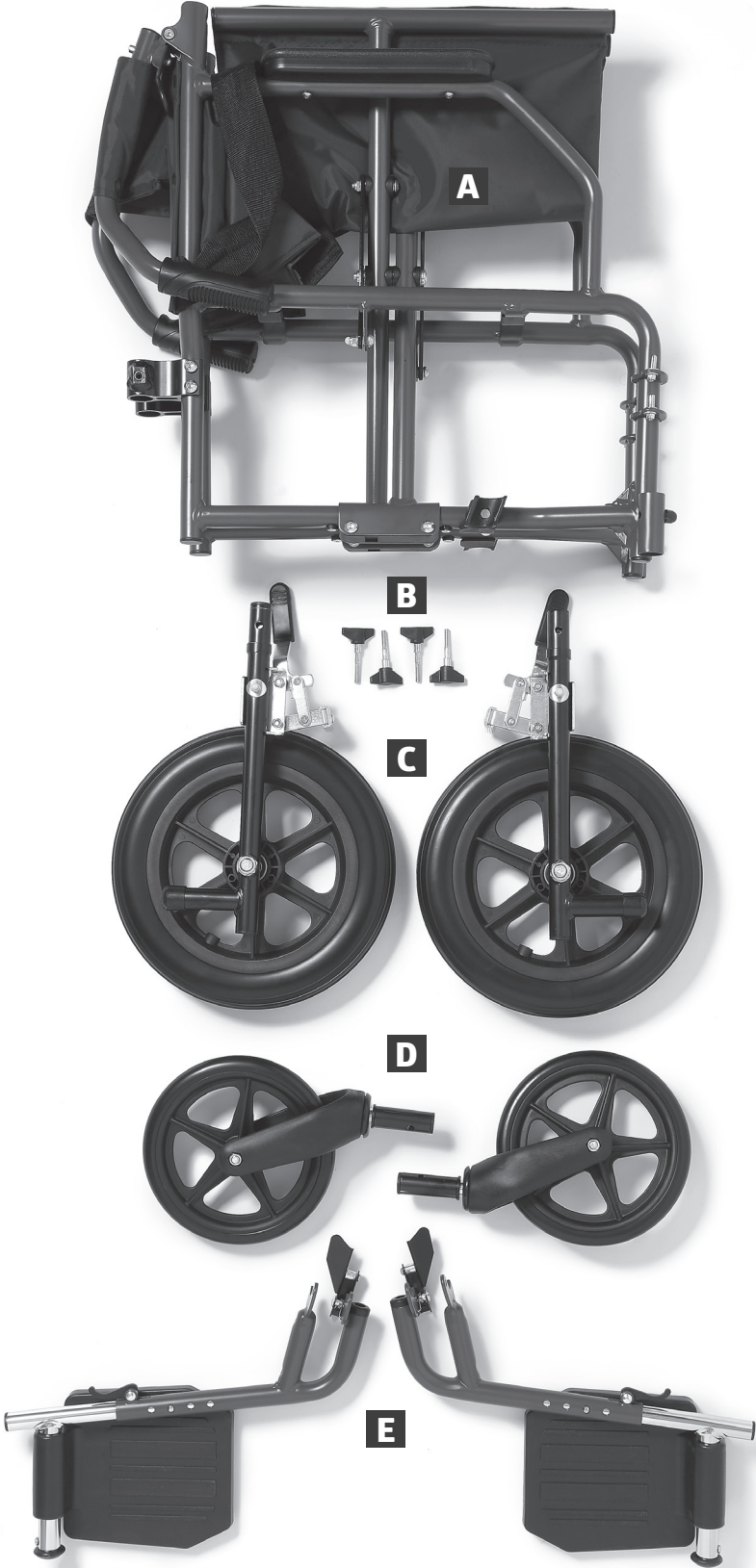
**F** Step-By-Step  
Assembly Guide

**G** General Warnings

## This guide applies to the following chair:

Item Number	Description	Color	Weight Capacity
MDS808210KDR	Transport Chair, Lightweight Steel, 19" x 16" (48.2 x 40.6 cm) Seat, Swing-Away Footrests, Desk Length Arms, and Stow-N-Go Clips	Grey & Burgundy	300 lb (136 kg)
MDS808200KDT	Transport Chair, Lightweight Steel, 19" x 16" (48.2 x 40.6 cm) Seat, Swing-Away Footrests, Desk Length Arms, and Stow-N-Go Clips	Grey & Teal	300 lb (136 kg)
MDS808200KWM	Transport Chair, Lightweight Steel, 19" x 16" (48.2 x 40.6 cm) Seat, Swing-Away Footrests, Desk Length Arms, and Stow-N-Go Clips	White & Light Tan	300 lb (136 kg)
MDS808200KNM	Transport Chair, Lightweight Steel, 19" x 16" (48.2 x 40.6 cm) Seat, Swing-Away Footrests, Desk Length Arms, and Stow-N-Go Clips	Navy & Blue	300 lb (136 kg)

**APPROVED**

By Ivy Hu at 1:29 pm, Aug 21, 2023

# First: Open the Chair

With the seat in an upright position, place hands on either side of the arms of the chair and gently pull outward.

As the chair begins to open, press down firmly on the outer edges of the seat with both hands, keeping fingers flat.

## IMPORTANT NOTE:

Proceed with caution when opening the chair as there are several areas around the moving parts that could potentially pinch fingers.

**APPROVED**

By Ivy Hu at 1:29 pm, Aug 21, 2023

# Next: Install the Wheels

## Front Wheels

Place the chair on its back.

Remove the black plastic plug from the outer tube and discard.

Unscrew the black plastic knob by turning it counterclockwise. Set aside.

**APPROVED**  
By Ivy Hu at 1:29 pm, Aug 21, 2023

Insert the wheel stem into the tube, making sure to align the hole in the wheel stem with the hole in the tube.

Re-insert the black plastic knob and turn it clockwise to tighten and secure the wheel to the chair.

**6** Repeat these steps on the other side.

**APPROVED**  
By Ivy Hu at 1:29 pm, Aug 21, 2023

## Rear Wheels

1

Place the chair on its front.

2

Unscrew the black knob from the chair base by turning it counterclockwise. Set aside.

**APPROVED**  
By Ivy Hu at 1:29 pm, Aug 21, 2023

**3**

Insert the wheel stem into the tube, making sure to align the hole in the wheel stem with the hole in the tube.

**4**

Re-insert the black plastic knob and turn it clockwise to tighten and secure the wheel to the chair.

**5** Repeat these steps on the other side.

**APPROVED**  
By Ivy Hu at 1:29 pm, Aug 21, 2023

## Engaging the Wheel Locks

The wheel locks are pre-installed on the rear wheels.

Push the lever towards the front of the chair to engage and lock the rear wheels.

Push the lever towards the back of the chair to unlock the rear wheels.

### IMPORTANT NOTE:

Engage both wheel locks before entering or exiting the transport chair, when leaning forward, or while in an elevator or wheelchair lift.

**APPROVED**

By Ivy Hu at 1:29 pm, Aug 21, 2023

## Next: Install the Footrests

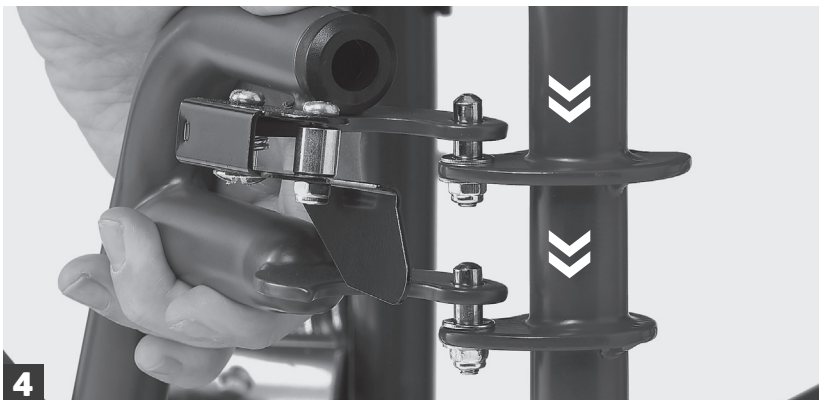
Engage the rear wheel locks by pushing the black lever towards the front of the chair.

Locate the two small silver pins on the chair above the front wheels.


Locate the two pre-drilled holes on the footrest.

Slide the footrest openings over the silver pins and place down.

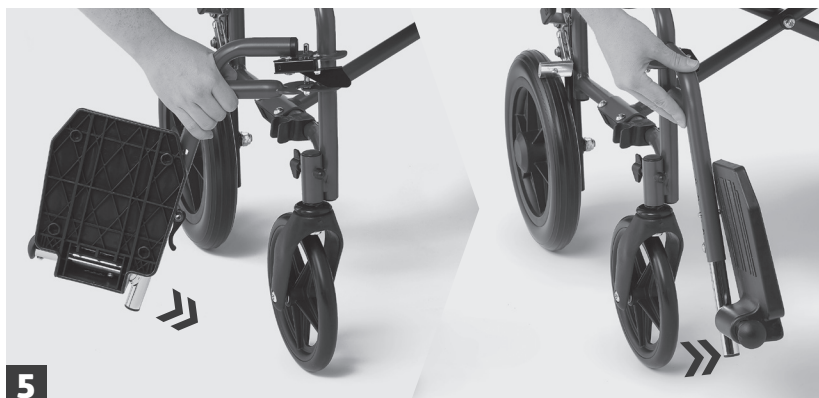**APPROVED**  
By Ivy Hu at 1:29 pm, Aug 21, 2023

Swing the footrest to the front of the chair until it locks into place.

Fold the foot plate down to use.

**7** Repeat these steps on the other side.

**IMPORTANT NOTE:**

If the foot plate is facing outwards when attached to the chair, See section How To adjust the footrest height.

**APPROVED**  
By Ivy Hu at 1:29 pm, Aug 21, 2023

# To Swing Away the Footrest:

**1** Fold the foot plate up to the vertical position.

Release the footrest lock by pushing the black lever forward.

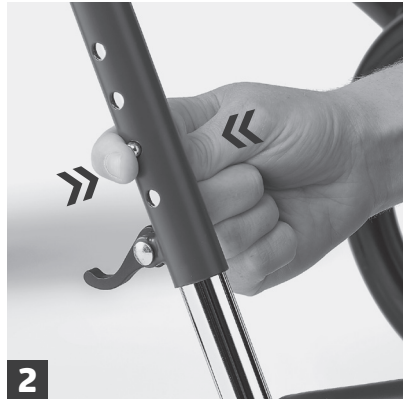
Snap the footrest into the Stow-N-Go Clips located behind the front wheel to secure it to the chair.

**APPROVED**  
By Ivy Hu at 1:29 pm, Aug 21, 2023

## To Adjust the Footrest Height:

Flip up the black tab on the back of the leg rest.

Squeeze the push buttons on the stem of the leg rest to adjust the footrest height up or down.

When the desired height is achieved, release the pressure on the push buttons. The push buttons click securely into pre-drilled holes.

Flip down the black tab on the back of the footrest.

### IMPORTANT NOTE:

If the foot plate is facing outwards when attached to the chair, follow instructions from figure 1 and figure 2, then rotate the foot plate so it is on the inside of the chair. Follow instructions for figure 3 and figure 4 to lock the footrest.

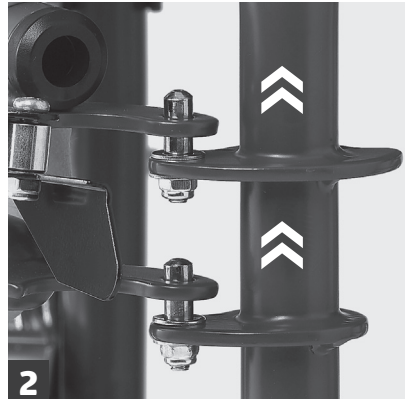
**APPROVED**

By Ivy Hu at 1:29 pm, Aug 21, 2023

## To Remove the Footrest From the Chair:

Disengage the tab lock.

Pull the footrest up and off of the silver pins.

## Finally: Unfold the Seat Back

Using the handles, push the seat back up until the release assembly spring buttons click fully into the folding mechanism on both sides. (You should be able to see the spring buttons fully engaged from the front of the seat back canes.)

Ensure the frame/handles and seat back are locked securely into place before using the chair.

**APPROVED**  
By Ivy Hu at 1:29 pm, Aug 21, 2023

# Close and Fold

Fold the footrest plates up to a vertical position, pull them back, and snap them into the Stow-N-Go holders on the side of the chair.

Turn the front wheels back. Unlock the back wheels.

Find the black release tabs on each side of the chair back canes and pull down. The seat back will collapse.

Using both hands, grasp the two fabric handles on the seat upholstery and pull up.

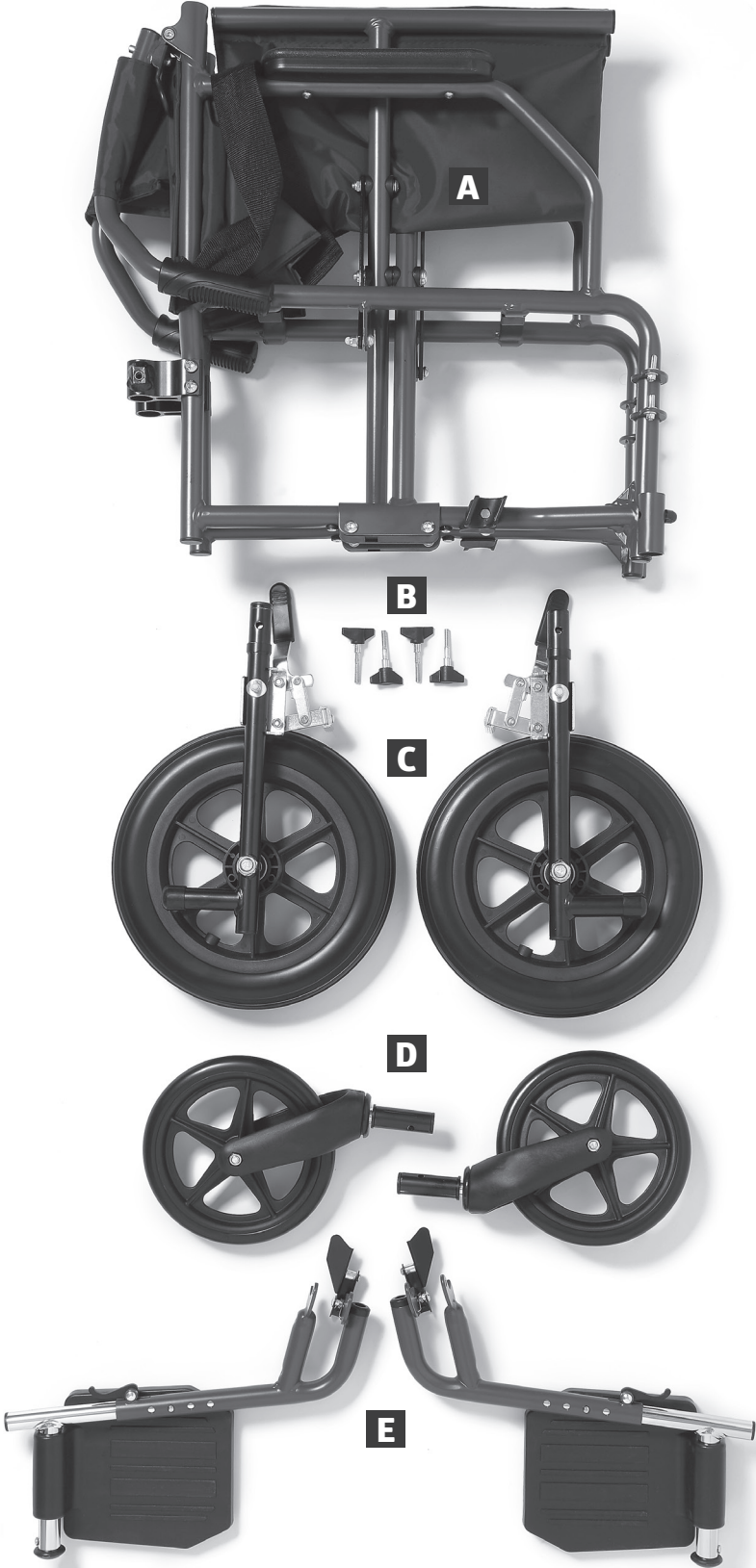
Now grasp the arms of the chair and push inward until it collapses further.

Chair folds down for easy storage and transport.

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**APPROVED**

By Ivy Hu at 1:29 pm, Aug 21, 2023

## **ENTRAPMENT, STRANGULATION, SUFFOCATION AND FALL HAZARDS**

Gaps in and around this product can entrap and kill people with Alzheimer's disease or dementia, or those who are sedated, confused, or frail, and are at increased risk of entrapment and strangulation. People attempting to climb over this product are at increased risk of injury or death from falls. Always make sure this product is properly secured to bed. If product can move away from bed or mattress, it can lead to entrapment and death.

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Microban® antimicrobial product protection is limited to the product itself and is not designed to protect users from disease causing microorganisms or as a substitute for normal cleaning practices.

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**APPROVED**

By Ivy Hu at 1:30 pm, Aug 21, 2023

We hope you enjoy your transport chair for years to come. In order to ensure your safety, it is necessary that you read and understand the following guidelines. Failure to do so may result in serious bodily injury and/or damage to the chair.

- » **DO NOT** operate the chair without first reading and understanding the instructions.
- » If you need assistance in understanding the warning, cautions, or instructions, **before use** contact a healthcare professional, durable medical dealer, or technical personnel, or contact Medline at 1-800-MEDLINE (633-5463) | 1-800-268-2848.
- » A caregiver must operate the chair at all times.

A qualified transport chair technician must perform any repairs to the chair, or serious injury may occur.

- » Technicians should review this manual before attempting to repair the chair.
- » Accessories, parts, or adapters must be authorized by Medline. Questions should be directed to 1-800-MEDLINE (633-5463) | 1-800-268-2848.

**APPROVED**

*By Ivy Hu at 1:30 pm, Aug 21, 2023*

# Usage

Transport chairs are ideal for airports, shopping centers, or other public places that require short-term transport from one location to another. **They are not intended for independent use by transport chair-bound individuals.**

This transport chair has not been approved for transport use in cars. Prior to arriving at your destination, always place the user in an approved vehicle seat and use approved restraints available from the auto industry.

- » **DO NOT** exceed the weight capacity for this transport chair, which includes the weight of the rider and items carried or attached to the transport chair.
- » **DO NOT** use the chair when the back upholstery is folded.
- » **DO NOT** use the transport chair while weight training. It has not been tested for such usage. If weight capacity is exceeded or shifted, a fall, tip over or loss of control may occur, which can result in bodily harm or product damage.
- » **DO NOT** stand on the frame of the chair.
- » **DO NOT** stand on or put pressure on the foot rests. Place the foot plates in an upright position before getting in and out of the chair.
- » **DO NOT** attempt to lift the transport chair by any detachable components.
- » **DO NOT** sit in the transport chair until the seat is completely open. Always keep hands and fingers away from moving parts.
- » **DO NOT** transfer in and out of the transport chair on your own unless you have sufficient body strength, balance, and agility and have been instructed in transfer techniques by a health professional. Before you move, turn both front casters parallel to the object you are transferring to. Every precaution should be taken to reduce the gap distance between the chair and the target seat or bed. Engage all wheel locks for maximum safety.

# Adjustments

**DO NOT** adjust the foot rests any closer than 2.5 inches (6.3 cm) from the floor.

**DO NOT** modify the transport chair. Examples include: drilling holes, grinding, welding parts, using non-authorized/non-approved parts. Such modifications may change the transport chair's structure and center of balance, which may increase the risk of serious injury to the user and void the warranty.

**APPROVED**

*By Ivy Hu at 1:30 pm, Aug 21, 2023*

# Environmental

Contact with water or excess moisture may cause the non-aluminum transport chairs to rust or corrode.

- » **DO NOT** use the transport chair in a shower, pool, or other body of water.
- » Avoid excess moisture (do not leave the chair in a damp bathroom while taking a shower).
- » Dry the chair immediately if exposed to moisture.

When cleaning the hand grips, use only a clean towel lightly dampened with cool water.

- » Ensure the grips are dry before use.
- » Use of soap or ammonia-based cleaning solutions may result in the hand grips sliding off.

If the chair is exposed to extreme temperatures, high humidity and/or becomes wet, check the hand grips to ensure they are not loose and do not twist on the handle.

Avoid storing or using the chair near open flames or combustible products.

# Safety Guidelines

Work with the transport chair user's healthcare professional or your institution to learn safe ways to assist within your capability and comfort level.

To prevent injury to yourself or the transport chair occupant, you should use good body mechanics:

- » Always tell the occupant what you intend to do so he/she is ready for your actions and can assist with maintaining a balanced center of gravity for stability.
- » Bend at the knees, not at the waist. Keep knees bent, not locked straight.
- » Use your legs for strength, not the weaker muscles of the back or arms.
- » **DO NOT** twist at the waist. Keep the torso facing the same direction as the hips and move the feet to turn.
- » Maintain a straight back. Hunching or rounding the shoulders can cause back strain.
- » If you do not feel comfortable with an activity you are requested to help with, you may decline to assist and should help find appropriate assistance for the occupant.

**APPROVED**  
By Ivy Hu at 1:30 pm, Aug 21, 2023

## Transferring into and out of the transport chair

- » Position the transport chair as close as possible to the patient.
- » Make sure both wheel locks are engaged.
- » Remove or swing away the foot rests that are in the transfer path.
- » Transfer the occupant as far back into the seat of the transport chair as possible.

## Reaching, Leaning or Bending— Forward, Sideways or Backwards

- » Make sure the front casters are pointing in a forward position. This extends the wheel base and will help stabilize the chair.
- » Position the chair as close as possible to the object.
- » **DO NOT** engage the wheel locks—if your weight shifts while reaching backwards it is better for the transport chair to roll than to tip.
- » **DO NOT** attempt to reach for objects if doing so means you must move forward in the seat any farther than the length of the arm rests.
- » **DO NOT** lean forward out of the transport chair any farther than the length of the arm rests.
- » **DO NOT** shift your weight or seat position toward the direction you are reaching as the chair may tip.
- » **DO NOT** attempt to reach for objects if you have to pick them up from the floor by reaching down between your knees.
- » **DO NOT** lean back over the top of the back upholstery to reach objects. Reach back only as far as your arm will extend without changing your position in the seat.

## Curbs, Steps and Stairs

Do not attempt to negotiate curbs or single steps without assistance. Ask your healthcare professional to issue instructions regarding safe transport chair use for curbs, steps, and stairs. Proper body mechanics and awareness of the center of gravity is crucial for these maneuvers. The step tubes at the back of the transport chair are utilized to tilt the transport chair backwards until the balance point is achieved and the front casters clear the curb/step. Gently lower the front wheels to the ground and slowly roll the transport chair forward until the rear wheels roll up over the curb/step.

Extreme caution is required if it is absolutely necessary to maneuver an occupied transport chair up or down stairs. Two assistants with adequate body strength and endurance must be present.

**DO NOT** lift the transport chair by any removable parts (i.e. swing-away foot rests). Make sure the hand grips are not loose. Techniques for going up or down steps should be received from a health care professional familiar with transport chair use and practiced before attempting to transport on stairs. If at all possible, an alternate means of transport up or down stairs is recommended.

**APPROVED**

*By Ivy Hu at 1:30 pm, Aug 21, 2023*

## Obstacles

Obstacles such as floor threshold strips, uneven floors, cords or items on the floor, or cracks in pavements can damage your transport chair and may cause loss of control or a tip over.

Be alert! Scan your path and avoid any items that may cause instability.

## Ramps and Inclines

The healthcare professional must know their capabilities and limitations in terms of strength and endurance before attempting to go up or down a ramp.

- » When going down an incline, the healthcare professional should turn the transport chair and go down backwards to avoid tipping the occupant out of the transport chair.

The transport chair is designed for use on firm, even surfaces, such as concrete, asphalt, and indoor floors and carpeting.

- » **DO NOT** operate the transport chair on streets or roadways.
- » **DO NOT** operate the transport chair on loose, slippery, or uneven surfaces, such as sand, gravel, hills, dirt roads, or ice.
- » **DO NOT** attempt to go up or down inclines that are covered with oil, water, ice, or any other potentially slippery substance.
- » **DO NOT** attempt to ride over curbs or obstacles. Doing so may cause your transport chair to tip over and cause bodily harm to you, or damage to the chair.
- » **DO NOT** use the transport chair on stairs or escalators.

When going down a ramp, the occupant should face forward and lean backward into the seat. Descent should always be done slowly. Remember that inclines will change the center of gravity.

- » **DO NOT** operate the transport chair on inclines greater than 10% (6 degrees) without assistance.
- » Use of a seat belt is advised to help maintain stability by minimizing movement within the transport chair seat. The use of anti-tip devices especially when maneuvering on ramps or inclines will also enhance safety.
- » Backpacks or other gear on the back of the transport chair will cause it to tip backward more easily. Training and practice negotiating ramps with a healthcare professional is recommended.
- » **DO NOT** change direction when going down an incline as it may cause instability.

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**APPROVED** recomienda usar  
By Ivy Hu at 1:30 pm, Aug 21, 2023

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**APPROVED**

**By Ivy Hu at 1:30 pm, Aug 21, 2023**

**Medline Industries, LP, Three Lakes Drive, Northfield, IL 60093 USA**

Medline United States  
1-800-MEDLINE (633-5463)  
medline.com | info@medline.com

**Medline Canada**

1-800-268-2848 | medline.ca | canada@medline.com

**Medline México**

01-800-831-0898 | medlinemexico.com | mexico@medline.com

Follow us  
S'avez-nous  
Síguenos

Some products may not be available for sale in Mexico or Canada. Please contact your Account Representative for additional information. We reserve the right to correct any errors that may occur within this manual. ©2023 Medline Industries, LP. Guardian and Medline are registered trademarks of Medline Industries, LP. Microban® is a registered trademark of Microban Products Company.

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