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Before use, it is necessary that you read and understand the following guidelines. Failure to do so may result in serious bodily injury and/or damage to the chair.

- » **DO NOT** operate the chair without first reading and understanding the instructions.
- » If you need assistance in understanding the warning, cautions, or instructions, before use contact a healthcare professional, durable medical dealer, or technical personnel, or contact Medline at 1-800-MEDLINE (633-5463) | 1-800-268-2848.
- » A caregiver must operate the chair at all times.

A qualified transport chair technician must perform any repairs to the chair, or serious injury may occur.

- » Technicians should review this manual before attempting to repair the chair.
- » Accessories, parts, or adapters must be authorized by Medline. Questions should be directed to 1-800-MEDLINE (633-5463) | 1-800-268-2848.

## Usage

Transport chairs are ideal for airports, shopping centers, or other public places that require short-term transport from one location to another. **They are not intended for independent use by transport chair-bound individuals.**

This transport chair has not been approved for transport use in cars. Prior to arriving at your destination, always place the user in an approved vehicle seat and use approved restraints available from the auto industry.

- » **DO NOT** exceed the weight capacity for this transport chair, which includes the weight of the rider and items carried or attached to the transport chair.
- » **DO NOT** use the chair when the back upholstery is folded.
- » **DO NOT** use the transport chair while weight training. It has not been tested for such usage. If weight capacity is exceeded or shifted, a fall, tip over or loss of control may occur, which can result in bodily harm or product damage.
- » **DO NOT** stand on the frame of the chair.
- » **DO NOT** stand on or put pressure on the foot rests. Place the foot plates in an upright position before getting in and out of the chair.
- » **DO NOT** attempt to lift the transport chair by any detachable components.
- » **DO NOT** sit in the transport chair until the seat is completely open. Always keep hands and fingers away from moving parts.
- » **DO NOT** transfer in and out of the transport chair on your own unless you have sufficient body strength, balance, and agility and have been instructed in transfer techniques by a health professional. Before you move, turn both front casters parallel to the object you are transferring to. Every precaution should be taken to reduce the gap distance between the chair and the target seat or bed. Engage all wheel locks for maximum safety.

## Adjustments

**DO NOT** adjust the foot rests any closer than 2.5 inches (6.3 cm) from the floor.

**DO NOT** modify the transport chair. Examples include: drilling holes, grinding, welding parts, using non-authorized/non-approved parts. Such modifications may change the transport chair's structure and center of balance, which may increase the risk of serious injury to the user and void the warranty.

## Environmental

Contact with water or excess moisture may cause the non-aluminum transport chairs to rust or corrode.

- » **DO NOT** use the transport chair in a shower, pool, or other body of water.
- » Avoid excess moisture (do not leave the chair in a damp bathroom while taking a shower).
- » Dry the chair immediately if exposed to moisture.

When cleaning the hand grips, use only a clean towel lightly dampened with cool water.

- » Ensure the grips are dry before use.
- » Use of soap or ammonia-based cleaning solutions may result in the hand grips sliding off.

If the chair is exposed to extreme temperatures, high humidity and/or becomes wet, check the hand grips to ensure they are not loose and do not twist on the handle.

Avoid storing or using the chair near open flames or combustible products.

## Safety Guidelines

Work with the transport chair user's healthcare professional or your institution to learn safe ways to assist within your capability and comfort level.

To prevent injury to yourself or the transport chair occupant, you should use good body mechanics:

- » Always tell the occupant what you intend to do so he/she is ready for your actions and can assist with maintaining a balanced center of gravity for stability.
- » Bend at the knees, not at the waist. Keep knees bent, not locked straight.
- » Use your legs for strength, not the weaker muscles of the back or arms.
- » **DO NOT** twist at the waist. Keep the torso facing the same direction as the hips and move the feet to turn.
- » Maintain a straight back. Hunching or rounding the shoulders can cause back strain.
- » If you do not feel comfortable with an activity you are requested to help with, you may decline to assist and should help find appropriate assistance for the occupant.

## Transferring into and out of the transport chair

- » Position the transport chair as close as possible to the patient.
- » Make sure both wheel locks are engaged.
- » Remove or swing away the foot rests that are in the transfer path.
- » Transfer the occupant as far back into the seat of the transport chair as possible.

## Reaching, Leaning or Bending— Forward, Sideways or Backwards

- » Make sure the front casters are pointing in a forward position. This extends the wheel base and will help stabilize the chair.
- » Position the chair as close as possible to the object.
- » **DO NOT** engage the wheel locks—if your weight shifts while reaching backwards it is better for the transport chair to roll than to tip.
- » **DO NOT** attempt to reach for objects if doing so means you must move forward in the seat any farther than the length of the arm rests.
- » **DO NOT** lean forward out of the transport chair any farther than the length of the arm rests.
- » **DO NOT** shift your weight or seat position toward the direction you are reaching as the chair may tip.
- » **DO NOT** attempt to reach for objects if you have to pick them up from the floor by reaching down between your knees.
- » **DO NOT** lean back over the top of the back upholstery to reach objects. Reach back only as far as your arm will extend without changing your position in the seat.

## Curbs, Steps and Stairs

Do not attempt to negotiate curbs or single steps without assistance. Ask your healthcare professional to issue instructions regarding safe transport chair use for curbs, steps, and stairs. Proper body mechanics and awareness of the center of gravity is crucial for these maneuvers. The step tubes at the back of the transport chair are utilized to tilt the transport chair backwards until the balance point is achieved and the front casters clear the curb/step. Gently lower the front wheels to the ground and slowly roll the transport chair forward until the rear wheels roll up over the curb/step.

Extreme caution is required if it is absolutely necessary to maneuver an occupied transport chair up or down stairs. Two assistants with adequate body strength and endurance must be present.

**DO NOT** lift the transport chair by any removable parts (i.e. swing-away foot rests). Make sure the hand grips are not loose. Techniques for going up or down steps should be received from a health care professional familiar with transport chair use and practiced before attempting to transport on stairs. If at all possible, an alternate means of transport up or down stairs is recommended.

## Obstacles

Obstacles such as floor threshold strips, uneven floors, cords or items on the floor, or cracks in pavements can damage your transport chair and may cause loss of control or a tip over.

Be alert! Scan your path and avoid any items that may cause instability.

## Ramps and Inclines

The healthcare professional must know their capabilities and limitations in terms of strength and endurance before attempting to go up or down a ramp.

- » When going down an incline, the healthcare professional should turn the transport chair and go down backwards to avoid tipping the occupant out of the transport chair.

The transport chair is designed for use on firm, even surfaces, such as concrete, asphalt, and indoor floors and carpeting.

- » **DO NOT** operate the transport chair on streets or roadways.
- » **DO NOT** operate the transport chair on loose, slippery, or uneven surfaces, such as sand, gravel, hills, dirt roads, or ice.
- » **DO NOT** attempt to go up or down inclines that are covered with oil, water, ice, or any other potentially slippery substance.
- » **DO NOT** attempt to ride over curbs or obstacles. Doing so may cause your transport chair to tip over and cause bodily harm to you, or damage to the chair.
- » **DO NOT** use the transport chair on stairs or escalators.

When going down a ramp, the occupant should face forward and lean backward into the seat. Descent should always be done slowly. Remember that inclines will change the center of gravity.

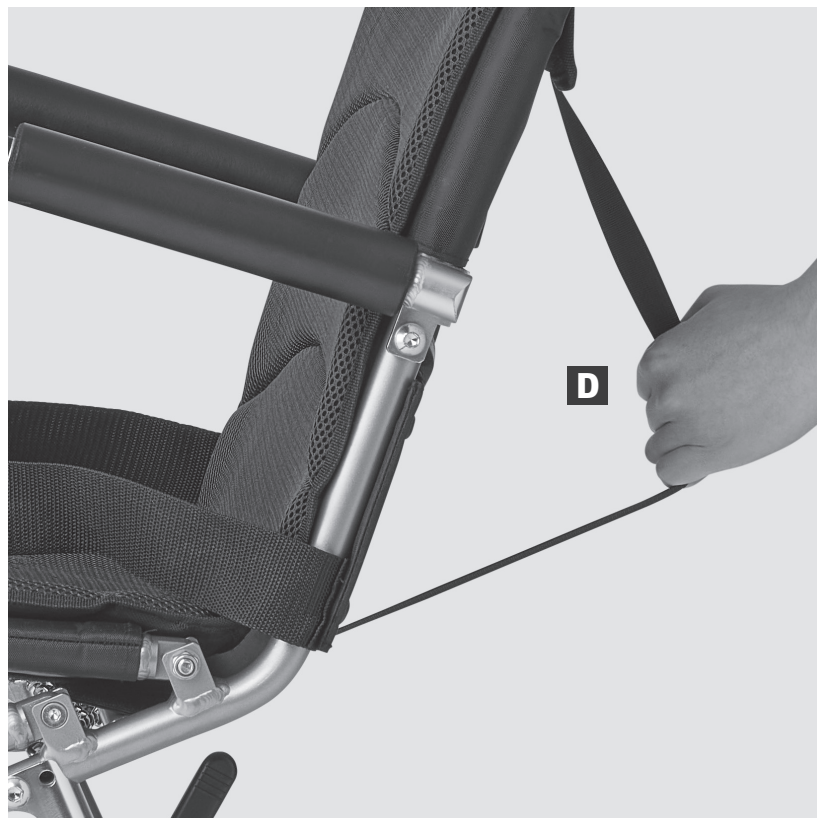
- » **DO NOT** operate the transport chair on inclines greater than 10% (6 degrees) without assistance.
- » Use of a seat belt is advised to help maintain stability by minimizing movement within the transport chair seat. The use of anti-tip devices especially when maneuvering on ramps or inclines will also enhance safety.
- » Backpacks or other gear on the back of the transport chair will cause it to tip backward more easily. Training and practice negotiating ramps with a healthcare professional is recommended.
- » **DO NOT** change direction when going down an incline as it may cause instability.

## What's Included:

- A** Chair Frame  
(with leg rests)
- B** 2 Hand Screws
- C** 2 Front Wheels


- D** Pull Strap
- E** User Manual

## This guide applies to the following chair:

Item Number	Description	Color	Weight Capacity
TCP198N22ST	Transport Chair, Folds Flat	Aluminum	250 lb (113 kg)
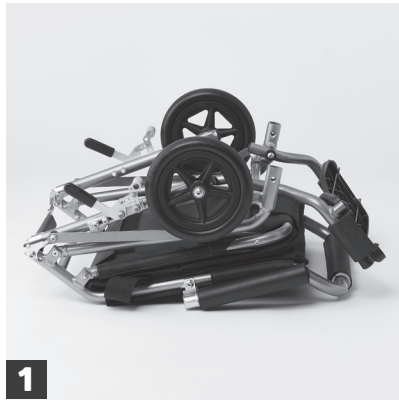
## First: Install the Wheels

**1** Lay the chair on its back.

**2** Push the two push buttons on the stem of a front wheel and insert the stem into the tube.

**3** Ensure the push buttons are aligned with the holes in the tube and pop back out.

**4** Insert the screw through the front of the frame tube.

**5** Reattach the black knob onto the end of the hand screw by turning clockwise until tight.

**6** Repeat this process for the other wheel.

## Next: Open the chair

**1** Flip the chair right side up, so that the wheels are on the floor.

**2** Pull up on the top support bar to begin unfolding the chair.

**3** While holding top support bar, pull up on the strap to allow chair to unlock into the open position.

**4** Fold down each arm rest.

## Engaging the Wheel Locks

**1** Pull the lever towards the back of the chair to engage and lock the rear wheels.

**2** Push the lever towards the front of the chair to unlock the rear wheels.

## Next: Adjust the Footrests

**1** Pull down on each footrest to open.

**2** Push up each footrest to close. Ensure the black platform is placed underneath the seat when folded up.

**3** Footrest can be adjusted by lifting the black foot platform upwards.

## Close and Fold

**1** Ensure the footrests are in the down position.

**2** Fold up each arm rest.

**3** Turn the front wheels back. Unlock the back wheels.

**4** Pull up on the release chain to begin to close the chair.

**5** Push down on the top bar until the chair is in the folded position.

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## Limited Warranty

Your Medline product is warranted to be free of defects in materials and workmanship for the duration of use by the original purchaser. This device was built to precise standards and carefully inspected prior to shipment. This limited warranty is an expression of our confidence in the materials and workmanship of our products and assures years of convenient, dependable service. In the event of a defect covered by this warranty, we will, at our option, repair or replace the device. This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters and grips, which are subject to normal wear and replacement.

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Some products may not be available for sale in Mexico or Canada. Please contact your Account Representative for additional information. We reserve the right to correct any errors that may occur within this manual. ©2025 Medline Industries, LP. Medline is a registered trademark of Medline Industries, LP.

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