GUARDIAN[®]

Family-owned. Trusted by
hospitals for over 50 years.

TUB GRAB BAR USER INSTRUCTIONS & WARRANTY

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REF G98006

**LIFETIME
LIMITED
WARRANTY**

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PLEASE READ CAREFULLY BEFORE INSTALLING ON TUB.

WARNING

Read carefully. Your personal safety will depend on following these instructions.

WARNING

The tub wall, both inside and out, should be clean of soap residues and free of any obstructions. This is essential if the clamping action is to work correctly.

WARNING

Ensure all surfaces and clamping pads are DRY and free of soapy residue. Clean and dry before each installation.

WARNING

On certain types of fiberglass or thin steel tubs, use extreme care not to over tighten. If over tightened, the clamp may crack the tub.

WARNING

Check for tightness and stability before each use.

WARNING

The Guardian® tub grab bar should rest directly on the top of the tub wall. If a shower door track interferes with this contact, the unit should not be used.

INSTALLATION INSTRUCTIONS

1. Open clamps by turning the adjustment knob counter-clockwise.
2. Place on tub with adjustment knob on outside of tub and secure tightly by turning adjustment knob clockwise.
3. Before using your Guardian® Tub Grab Bar, always test for stability by vigorously attempting to lift and move unit sideways. If unit is clamped correctly, it should be extremely stable and rigid, even when substantial forces are applied.

CLEANING

Soap and water is the recommended cleaning method. Avoid the use of harsh cleaners.

Maximum User Weight: 250 lbs. (113.5 kg) Not Made With Natural Rubber Latex.

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LIMITED WARRANTY

Your Medline product is warranted to be free of defects in materials and workmanship for the duration of use by the original purchaser. This device was built to precise standards and carefully inspected prior to shipment. This limited warranty is an expression of our confidence in the materials and workmanship of our products and assures years of convenient, dependable service. In the event of a defect covered by this warranty, we will, at our option, repair or replace the device. This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters and grips, which are subject to normal wear and replacement.

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GUARDIAN[®]

Medline Industries, LP,
Three Lakes Drive,
Northfield, IL 60093 USA

Medline United States
1-800-MEDLINE (633-5463)
www.medline.com | info@medline.com

Medline Canada
1-800-396-6996
www.medline.ca | Canada@medline.com

Medline Mexico
01-800-831-0898
www.medlinemexico.com | mexico@medline.com

Some products may not be available for sale in Mexico or Canada. Please contact your Account Representative for additional information. We reserve the right to correct any errors that may occur within this manual. ©2018 Medline industries, LP. Guardian is a registered trademark of Medline Industries, LP.

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