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## Safety Instructions

To ensure your safety in using the Medline rollator, the following safety information and all instructions must be followed.

### GENERAL WARNINGS AND CAUTIONS:

- **DO NOT** use this product without first reading and understanding the instructions contained in this manual. If you are unable to understand the warnings, cautions or instructions, contact a healthcare professional or dealer before use. Failure to do so may result in bodily injury or damage to the device.
- **DO NOT** self-propel the rollator while seated. Rollators are intended for individual use only and are **NOT TO BE USED AS A WHEELCHAIR**. Serious injury to the user and/or damage to the rollator may result from improper use.
- **DO NOT** use the rollator to walk backwards, down gradients or to climb stairs. Avoid curbs and other obstacles as serious risk of fall or injury may occur.
- All wheels must be in contact with the floor and/or ground at **ALL TIMES**.
- **DO NOT** exceed the maximum weight capacity (including bag contents) or serious injury could result. Maximum weight capacity: 300 lb (136 kg). Place items completely in the bag so they do not protrude. Excess weight may cause the rollator to tip, resulting in bodily injury or damage to the rollator. The bag has a weight limitation of 5 lb (2.2 kg).
- **DO NOT** perform any adjustments to the rollator while it is in use. Before each use make sure the rollator frame is fully and securely open. Observe the minimum insertion depth of the push handles. Only use accessories and replacement parts authorized by Medline Industries, LP.

### SEAT WARNINGS:

- The Brakes **MUST** be in the locked position BEFORE sitting on the seat.
- **DO NOT** attempt to move the rollator while you or anyone is sitting in the seat.
- **DO NOT** sit on the rollator while it is being loaded into a vehicle or while it is being transported in a vehicle.
- **DO NOT** use the seat to transport people or objects.
- **DO NOT** sit on the seat when the rollator is on an incline.
- The rollator was designed to be used as a walking aid. **DO NOT** use the rollator in a manner that could cause personal injury such as a scooter, a knee walker or a jumping/hopping aid.
- **DO NOT** stick fingers or hands near the seat slide, folding mechanism or any moving parts. Use caution when folding and unfolding the rollator. Make sure that clothing and body parts do not get pinched between moving parts.
- **CAUTION:** The backrest is **NOT** intended to support full body weight. Avoid leaning while sitting as this may result in a fall.
- Use **EXTREME** caution when shifting weight while seated, as this may cause the rollator to tip.
- Check that the rollator is fully intact each time before using. Check that the brakes are in good working order prior to sitting.
- **DO NOT** "pop" or suddenly drop body weight onto the rollator seat. Ease gently onto the seat when attempting to sit.

### BRAKE WARNINGS:

- Check the brakes for proper operation prior to each use. Make sure that all parts are secure and that the moving parts are in good working order.
- **DO NOT** use the rollator to provide assistance to the user when rising from a lower position. The rollator may move even if the brakes are in the locked position and result in a fall.

- **DO NOT** push the rollator while the brakes are locked. Even if the brakes are locked it may not prevent the rollator from moving if pushed on certain surfaces.

### ENVIRONMENTAL WARNINGS:

- Contact with water or excess moisture may cause parts to rust or corrode.
- **DO NOT** use the rollator in a shower, pool or other body of water. Avoid excess moisture (do not leave the rollator in the bathroom while taking a shower).
- If the rollator is exposed to extreme temperatures, high humidity and/or becomes wet, check parts are in good working order and that the handgrips are not loose and cannot be twisted.
- **DO NOT** use or store the rollator near open flames or combustible materials.
- The surfaces of the rollator can heat up when exposed to direct sunlight or used on hot surfaces that could result in product damage and/or personal injury.

## Contents

Examine all parts for shipping damage. **DO NOT** use the rollator if there is apparent damage or if any parts (other than the bag) are missing. If there are missing or damaged parts, please call Medline Customer Care toll free at 1-800-MEDLINE/Medline Canada 1-800-268-2848.

After receiving the goods, please check the contents for completeness right away. The content consists of:

A: 1-Base

B: 2-Front casters

C: 2-Rear wheel assemblies with  
2- Brake assemblies attached

D: 2-Handles with armrests

E: 1-Backrest

F: 1-Bag

G: 2-Black curved washers

H: 4-Circular knobs

I: 2 - 1.5 in (38mm), Hex screws  
with thread glue

J: 4-5 point knobs

K: 4-Clear circular washers

L: 6-1.375 in (35mm),  
Hex screws with no thread glue

M: 2-C-clips

N: Cane holder

O: Seat

P: Handhold

Figure 1

## Summary

Product overview (Fig. 1)

## Assembly/adjustments

### 1. Locate Frame Base

- Position the rollator in the travel direction.
  - Press the frame tubes of the seat downward to unfold the rollator (Fig. 2). Make sure the tubes are aligned properly and snap into place.
- CAUTION:** Keep hands on the top of the seat and away from moving parts.

Figure 2

### 2. Attach 2 front wheels

- Insert front wheel assembly (Part B) into frame.
- Locate 1 - 1.5 in (38 mm) hex screw with blue thread glue (part I), curved black washer (part G), and a round knob (part H).
- Insert hex screw from inside the frame through frame and wheel assembly (Fig. 3).
- Place black curved washer on the outside with the curved side against the frame. Tighten round knob to the screw until secure.

Figure 3

### 3. Attach rear wheels

- Insert rear wheel assembly (part C) into frame until the hole on the assembly lines up with the hole on the frame. Make sure that the rear wheels and brake shoes are on the outside of the rollator (Fig. 4).
- Insert 1 - 1.375 in (35 mm) hex screw without thread glue (part L) from the inner side of the frame through the frame and wheel assembly.
- Place clear washer (part K) on screw on the outer side of the frame and use a 5 point knob (part J) to tighten until secure.

Figure 4

### 4. Attach handle assembly

- Insert universal handles into the frame with the hole by the forearm pads on the front of the frame (Fig. 5).
- Push down to lowest position.  
(Do not secure yet as it will make attaching the brake assembly easier.)

Figure 5

### 5. Attaching brake assembly and handle

- Insert brake assembly into the handle assembly.
- Line up the hole, in the desired position, on the brake assembly with the hole on the handle assembly.
- Insert C-clip (part M) from the top and push through both holes (Fig. 6).

Figure 6

### 6. Adjusting handle height

- Adjust the handles to desired height, ensuring the hole on the handle is lined up with the hole on the frame (Fig. 7).
- Insert 1 - 1.375 in (35 mm) hex screw without thread glue (part L) through the holes from the inner side of the frame.
- Attach clear washer (part K) to hex screw and use 5-point knob (part J) to tighten until secure.
- Handle height **MUST** be the same on both sides.
- Hand tighten only.

Figure 7

### 7. Attaching the backrest

- Attach brackets on the ends of the backrest (part E) to frame so that the hex opening is on the inside of the frame (Fig. 8).
- Insert 1 - 1.375 in (35 mm) hex screw without thread glue (part L) through both holes.
- Use the round knob (part H) to tighten.
- Before completely securing adjust to desired height and then secure the knob.

Figure 8

### 8. Attaching the accessories

- The rollator is unfolded. Activate the locking brakes.
- Position yourself against the travel direction in front of the rollator.
- The bag can be inserted from above in the guide rails on the left and right on the rollator (Fig. 9).
- Attach the cane holder base by pressing the U shaped clip over the button on the frame.
- To use place cane tip in the base of the cane holder and secure using the hook and loop strap higher up on the frame.

Figure 9

## Operating Instructions

### Activating the brakes

#### Service brake

To activate the service brake, pull both handbrake levers on the push handles upward (Fig. 10). Service and locking brake act on the rear wheels.

Figure 10

## Locking brake

- Press down on both brake levers to activate the locking device (Fig. 11).
- To release the locking brake pull both brake levers back into middle position.
- Loosen the locknut (Fig. 12) so the adjustment nut can turn.
- Now turn the wide adjustment nut further inwards or outwards to increase or decrease the braking effect.
- Retighten the locknut.
- Repeat the procedure on the other brake if necessary.
- Check the brakes for proper operation. Make sure that all parts are secure and that the moving parts are in good working order prior to each use.

Figure 11

## Tipping aid

The tipping aid is intended to make it easier for users to overcome curbs and small obstacles.

Step on the tipping aid with your foot and at the same time pull on the push handles. The front wheels lift slightly off the ground and with a little force you can push the rollator slightly past the obstacle.

Figure 12

## Folding and unfolding the assembled rollator

### Unfolding the Rollator

- Position the rollator in the travel direction.
- Release the locking brakes.
- Press the frame tubes of the seat downward to unfold the rollator. Make sure the tubes are aligned properly and snap into place.

**CAUTION:** Keep hands on the top of the seat and away from moving parts.

### Folding the Rollator

- Activate the locking brakes.
- Hold the rollator firmly with one hand on the handle. With your other hand, pull the grip on the seat upward.
- When doing so, you must overcome the resistance of the locking device.

## Walking

- Use on a level, firm surface
- Familiarize yourself with the function of the brakes
- Place yourself in the rollator so your ankles are at the height of the rear wheels.
- Stand with good posture
- Don't slouch or lean onto the push handles
- Grasp the push handles with both hands and your arms bent at a slight angle
- When walking, try to stay as close as possible to the push handles. Try to walk in the rollator and not behind it. Take small steps and DO NOT push the rollator too far ahead.
- Avoid having to lift the rollator over obstacles. Use ramps and go around obstacles whenever possible. Use assistance, if avoiding obstacles is not possible.

## Using the Seat

- Lock Parking brakes **BEFORE** sitting on the seat
- Use extreme caution when maneuvering to sit or to stand
- Turn your body so that push handles are at your sides and you are facing away from the backrest
- Carefully lower onto the seat as the rollator may move even if the brakes are in the locked position
- When rising to a standing position, use extreme caution to not push the rollator away
- Brakes **MUST** remain locked while seated

## Lifting points

You can lift the rollator on the grips or frame sections, in order to carry it over obstacles or steps.

## Technical Data/Specification

- Type: Upright Rollator
- Height (height of push handles): 41 in - 45.9 in
- Seat height: 21.6 in
- Rollator weight: 19.14 lb (8.68 kg)
- Weight Capacity of rollator: 300 lb (136 kg)
- Weight Capacity of bag: 5 lb (2.2 kg)

Periodic visual inspection of this product is recommended to ensure that all parts and hardware are secure, that components are in good working order and not cracked, broken or fractured, and that there are no obstructions or impediments to normal, safe operation. If any part or mechanism appears to be unsafe or damaged, do not use the product. If you experience problems, please contact Medline at 1-800-MEDLINE (633-5463) | Medline Canada 1-800-268-2848.

## Cleaning instructions:

- The frame of the rollator can be cleaned with a moist cloth with mild detergent
- The wheels can be cleaned with a wet synthetic fiber brush (**DO NOT** use wire brush)
- Push handles, seat and backrests can be wiped down using mild detergent
- **DO NOT** use aggressive or caustic detergents to clean the rollator
- Periodically wipe down the rollator with a soft cloth and non-abrasive cleaner
- Dry completely before use

## Brake Inspection and Adjustment:

- **DO NOT** use the rollator if the brakes are not functioning properly
- Make sure your brakes are operating properly before **EACH** use
- If the brakes are not functioning properly, discontinue use immediately and **DO NOT** use the rollator.
- Contact Medline customer care at 1-800-MEDLINE, if assistance is required

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## LIMITED WARRANTY

Your Medline product is warranted to be free of defects in materials and workmanship for the duration of use by the original purchaser. This device was built to precise standards and carefully inspected prior to shipment. This limited warranty is an expression of our confidence in the materials and workmanship of our products and assures years of convenient, dependable service. In the event of a defect covered by this warranty, we will, at our option, repair or replace the device. This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters and grips, which are subject to normal wear and replacement.

Limited lifetime Warranty on Frame, 6 months on: HAND BRAKES / BRAKE CABLES / PUSH-BUTTONS / BACKREST / BRAKES / UPHOLSTERY / WHEELS (FRONT AND REAR) / MEDLINE BAG  
ALL OTHER Replaceable PARTS.

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01-800-831-0898 | medlinemexico.com | mexico@medline.com

Follow us | Suivez-nous | Síguenos    

Some products may not be available for sale in Mexico or Canada. Please contact your Account Representative for additional information. We reserve the right to correct any errors that may occur within this manual. ©2023 Medline Industries, LP.

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©2023 Medline Industries, LP. All rights reserved/tous droits réservés/todos los derechos reservados. Medline is a registered trademark of/est une marque déposée de/ es una marca comercial registrada de Medline Industries, LP. Three Lakes Drive, Northfield, IL 60093 USA.