# GUARDIAN<sup>®</sup>

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REF G3-300BAX1

# General Warnings

To ensure the safe assembly and operation of this product, these instructions must be followed. **FAILURE TO FOLLOW INSTRUCTIONS MAY RESULT IN SERIOUS INJURY TO THE USER OR DAMAGE TO THE PRODUCT.** If you are unable to understand these warnings, cautions and/or instructions, contact a healthcare professional, dealer or Medline before attempting to use this product.

- DO NOT assemble and use this product without first reading and understanding this user's guide, otherwise injury or damage may occur.
- DO NOT use if components are missing or are damaged. Contact Medline at 1-800-MEDLINE (633-5463) | 1-800-268-2848.
- DO NOT exceed weight capacity of 600 lb (272 kg).
- DO NOT make any adjustments while using the product.
- DO NOT stand on the product.
- DO NOT attempt to use the arm to fully support full body weight.

## Cautions

Users with limited physical strength and/or mobility should be supervised and/or assisted while using the commode. Prior to use, ensure the following:

1. All push buttons are fully protruded and are properly engaged.
2. There are no loose, broken or missing components and ensure all parts are secure.
3. When using the backrest, ensure the product remains balanced. Excessive twisting or turning while seated on the bench may cause loss of balance and/or damage to the product.
4. All four legs touch the floor simultaneously at all times.
5. The tips on the leg extensions do not contain any rips, wear or are missing. Immediately replace missing or damaged tips.


# Assembly and Use

Frame is shipped partially assembled. If exposed to cold temperatures, allow contents to come to room temperature before completing assembly. Remove contents from box, making sure you have all parts described here:

- Backrest Assembly (1)
- Armrest (1)
- Leg Extensions with Suction Tips (4)
- Leg Extensions with Tips (2)

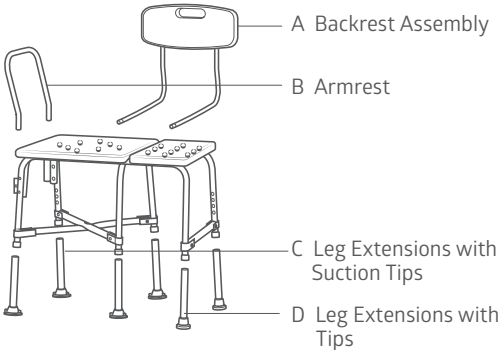Figure 1

1. Insert a leg extension with suction tip on all four legs of the bench. Ensure the legs are adjusted to the same height. (Figure 2)

Figure 2

2. Insert a leg extension with tip on the side legs of the bench. Ensure the legs are adjusted to the same height. (Figure 3)

Figure 3

3. Insert the armrest onto the bench by depressing the push buttons and inserting them into the armrest holsters. Ensure the curved bar extends outward. (Figure 4)

Figure 4

4. Insert the backrest assembly onto the bench by depressing the push buttons and inserting them into the backrest holsters under the seat. (Figure 5)

Figure 5

Note: Ensure all push buttons are fully engaged and protruding out the holes.

## Adjusting leg height

Before using the bench, make sure all legs are set to the same preferred height.

1. To adjust leg height, depress the push button and slide each leg to desired height by aligning with proper holes and letting the pushbutton snap into the correct hole to lock leg in place.
2. Make sure legs are locked into place by pushing down and pulling up on each leg.

# Maintenance

Periodic visual inspection of this bench is recommended to ensure that all parts and hardware are secure, that components are in good working order and are not worn, torn, frayed, or loose, and that there are no obstructions or impediments to normal, safe operation. If any part of mechanism appears to be unsafe or damaged, do not use the product. If you experience problems, please contact Medline at 1-800-MEDLINE | 1-800-268-2848.

# Cleaning

1. Clean with regular household cleaning products. Use a non-abrasive cleaner with warm water. Wipe after cleaning.
2. DO NOT use abrasive powders, cleaners or scouring pads.

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## Limited Warranty

Your Medline product is warranted to be free of defects in materials and workmanship for the duration of use by the original purchaser. This device was built to precise standards and carefully inspected prior to shipment. This limited warranty is an expression of our confidence in the materials and workmanship of our products and assures years of convenient, dependable service. In the event of a defect covered by this warranty, we will, at our option, repair or replace the device. This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as tip accessories, casters and grips, which are subject to normal wear and replacement.

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Some products may not be available for sale in Mexico or Canada. Please contact your Account Representative for additional information. We reserve the right to correct any errors that may occur within this manual. ©2022 Medline Industries, LP. Guardian and Medline are registered trademarks of Medline Industries, LP.

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