GUARDIAN[®]

User's Guide

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REF G1-301PSEAT

For use with G1-301DPX1:

1. Align the padded toilet seat clamps onto the rear frame. (All clamps should sit outside the pail brackets).
2. Push down on seat until clamps snap firmly into place.
3. To remove, grasp the back of the seat and lift.

For use with G1-502PWX1:

The two front clips of seat must be removed.

To remove:

1. Use a Phillips screwdriver to remove the clips from the front seat.
2. Align the padded toilet seat clamps onto the rear frame.
(Both clamps should sit outside the pail brackets).
3. Push down on seat until clamps snap firmly into place.
4. To remove, grasp the back of the seat and lift.

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Limited Warranty

Your Medline product is warranted to be free of defects in materials and workmanship for the duration of use by the original purchaser. This device was built to precise standards and carefully inspected prior to shipment. This limited warranty is an expression of our confidence in the materials and workmanship of our products and assures years of convenient, dependable service. In the event of a defect covered by this warranty, we will, at our option, repair or replace the device. This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters and grips, which are subject to normal wear and replacement.

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Some products may not be available for sale in Mexico or Canada. Please contact your Account Representative for additional information. We reserve the right to correct any errors that may occur within this manual. ©2021 Medline Industries, LP. Guardian and Medline are registered trademarks of Medline Industries, LP.

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**Medline Industries, LP, Three Lakes Drive,
Northfield, IL 60093 USA.**

Medline United States
1-800-MEDLINE (633-5463)
medline.com | info@medline.com

Medline Canada

1-800-268-2848 | medline.ca | canada@medline.com

Medline México

01-800-831-0898 | medlinemexico.com | mexico@medline.com

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