GUARDIAN[®]

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STEEL DROP-ARM COMMUNE

USER INSTRUCTIONS AND WARRANTY
Weight capacity 300 lbs

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REF G1-303DWX1

MAINTENANCE

Periodic visual inspection of this commode is recommended to ensure that all parts and hardware are secure, that components are in good working order and not worn, torn, frayed, or loose, and that there are no obstructions or impediments to normal, safe operation. If any part or mechanism appears to be unsafe or damaged, do not use the product. If you experience problems, please contact Medline at 1-800-MEDLINE.

ASSEMBLY and USE:

Frame is shipped pre-assembled. If exposed to cold temperatures, allow contents to come to room temperature before completing assembly.

1. Remove contents from box, making sure you have all parts described here. Locate backrest bar and snap into frame at the rear of the commode.
2. Align the toilet seat clamps onto the back rail of the frame. (Both clamps should sit outside the pail brackets). Push down on seat until clamps snap firmly into place. Repeat the same process to install the lid.
3. To drop the arms for access, depress the release button and allow armrest to gently drop. (Figure 1)
4. When removing the seat or lid, always remove lid first and then the seat. To remove, grasp the back of the lid and/or seat and lift.

Figure 1

5. Install commode bucket by dropping into bucket frame.
6. Remove bucket by lifting up out of the frame.
7. The commode is equipped with four locking wheels. To engage the lock, press down the lever on each of the wheels. To unlock, flip up the lever on each of the wheels. Ensure that all wheels are properly locked during use.

Adjusting Leg Height

1. Before using the commode, make sure all legs are set to the same preferred height.
2. To adjust leg height, depress pushbutton and slide each leg to desired height by aligning with proper holes and letting the pushbutton snap into the correct hole to lock leg in place. (Figure 2)
3. Make sure legs are locked into place by pushing down and pulling up on each leg.

Figure 2

CAUTIONS:

1. Toilet seat must be in down position before use. Failure to do so may result in possible injury or breaking the commode.
2. Do NOT assemble and use commode without first reading and understanding this assembly and instruction sheet. If you cannot understand these instructions, contact a healthcare dealer or technical professional before attempting to install. Otherwise, injury or damage may occur.
3. Inspect all parts for possible damage before assembling. If there is any damage - DO NOT USE. Contact dealer/carrier for further instructions.
4. Make sure that the height adjusting pushbuttons fully extend through the same corresponding hole of each leg extension. This ensures that the leg extensions are securely locked in place at an even height.
5. Make sure all screws, nuts and/or bolts are tightened.

6. Make sure the wheels and wheel locks are in place on the leg extensions and not worn, broken or missing. Replace any missing or damaged parts before use.
7. Wheels of all four leg extensions **MUST** touch the floor simultaneously at all times. Always ensure the wheel locks are properly engaged during use.
8. Users with limited physical strength should be supervised or assisted while using commode.
9. Use the arm supports only for assistance. **DO NOT** attempt to use the arms to support full body weight.
10. Arms should be unlatched and dropped to assist in transfer on or off the device.
11. **NEVER** stand on commode.

NOTE: To maintain appearance and promote good hygiene, clean periodically with mild soap and rinse with warm water. Wipe bath seat dry after each use.

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LIMITED WARRANTY

Your Medline product is warranted to be free of defects in materials and workmanship for the duration of use by the original purchaser. This device was built to precise standards and carefully inspected prior to shipment. This limited warranty is an expression of our confidence in the materials and workmanship of our products and assures years of convenient, dependable service. In the event of a defect covered by this warranty, we will, at our option, repair or replace the device. This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters and grips, which are subject to normal wear and replacement.

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GUARDIAN®

Medline Industries, LP,
Three Lakes Drive,
Northfield, IL 60093 USA.

Medline United States
1-800-MEDLINE (633-5463)
www.medline.com
info@medline.com

Medline Canada
1-800-396-6996
www.medline.ca
Canada@medline.com

Medline Mexico
01-800-831-0898
medlinemexico.com
mexico@medline.com

Some products may not be available for sale in Mexico or Canada. Please contact your Account Representative for additional information. We reserve the right to correct any errors that may occur within this manual. ©2019 Medline Industries, LP. Guardian and Medline are registered trademarks of Medline Industries, LP.

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