GUARDIAN[®]

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By Ivy Hu at 1:32 pm, Apr 14, 2023

Safety Instructions

To ensure your safety in using the Medline translator, the following safety information and all instructions must be followed.

General Warnings and Cautions:

- ⚠️ **DO NOT use this product without first reading and understanding the instructions contained in this manual.** If you are unable to understand the warnings, cautions or instructions, contact a healthcare professional or dealer before use. Failure to do so may result in bodily injury or damage to the device.
- ✓ The translator was designed and intended to be used as a walking aid and transport chair.
- ⚠️ The translator is **NOT** intended to be used as a full time wheelchair and/or transport chair replacement.
- ⚠️ **DO NOT** self-propel the translator while seated. Serious injury to the user and/or damage to the translator may result from improper use.
- ✓ A Caregiver/Transporter must be present at all times while the translator is in use for transport. Never leave the occupant unattended. This translator design is not intended to allow a patient to maneuver the translator.
- ⚠️ **DO NOT** use the translator to walk backwards, down gradients or to climb stairs or escalators. Avoid curbs and other obstacles as serious risk of fall or injury may occur.
- ✓ All wheels must be in contact with the floor and/or ground at **ALL TIMES**.
- ⚠️ **DO NOT** exceed the maximum weight capacity (including bag contents) or serious injury could result:
Maximum weight capacity: 300 lb (136 kg)
 - Place items completely in the bag/basket so they do not protrude. Excess weight may cause the translator to tip, resulting in bodily injury or damage to the translator.
 - The bag/basket has a weight limitation of 5 lb (2.2 kg).
- ⚠️ **DO NOT** perform any adjustments to the translator while it is in use.
- ✓ Before each use make sure the translator frame is fully and securely open.
- ✓ Observe the minimum insertion depth of the push handles.
- ✓ Only use accessories and replacement parts authorized by Medline Industries, LP.
- ⚠️ **DO NOT** attempt to lift the transport chair by any detachable components.

Seat Warnings:

- ⚠️ The Brakes **MUST** be in the locked position **BEFORE** sitting on the seat.
- ⚠️ **DO NOT** sit on the Translator while it is being loaded into a vehicle or while it is being transported in a vehicle.
- ⚠️ **DO NOT** sit on the seat when the Translator is on an incline.
- ⚠️ **DO NOT** use the Translator in a manner that could cause personal injury; such as a scooter, a knee walker or a jumping/hopping aid.
- ⚠️ **DO NOT** stick fingers or hands near the seat slide, folding mechanism or any moving parts. Use caution when folding and unfolding the Translator. Make sure that clothing and body parts do not get pinched between moving parts.
- ⚠️ **CAUTION:** The backrest is **NOT** intended to support full body weight. Avoid leaning while sitting as this may result in a fall.
- ✓ Use **EXTREME** caution when shifting weight while seated, as this may cause the Translator to tip.
- ✓ Check that the Translator is fully intact each time before using. Check that the brakes are in good working order prior to sitting.
- ⚠️ **DO NOT** use the hand brakes to stop the translator from moving. **DO NOT** use the hand brakes to slow your descent down an incline.
- ⚠️ **DO NOT** attempt to reach for objects if doing so means you must move forward in the seat.
- ⚠️ **DO NOT** attempt to reach objects if you have to pick them up from the floor by reaching down between your knees.

- ⚠ **DO NOT** shift your weight or sitting position toward the direction you are reaching as the translator may tip over.
- ⚠ **DO NOT** attempt to lean over the top of the back upholstery to reach objects.
Before leaning or reaching forward, make sure to rotate the casters straight ahead toward the front of the translator and engage the brakes to prevent the translator from tipping or moving.
- ⚠ **DO NOT** tip the translator without assistance. **DO NOT** lean the translator back since this can cause the translator to tip over. If trying to reach objects, reach back only as far as your arm will extend without altering your seating position.
- ✓ Any person assisting the rider in maneuvering or lifting of the translator must have sufficient strength and be able to see obstacles in the path of travel. When learning a new assistance technique, have an experienced assistant help you before attempting it alone. Remember to use good body mechanics.
- ⚠ **DO NOT** use the translator while weight training. Medline translators have not been tested for any type of weight training. If the weight capacity is exceeded, risk of fall, tip-over or loss of control may occur which can result in bodily injury or product damage.
- ⚠ **DO NOT** stand or put pressure on the footrests. When getting in or out of the Translator, make sure the footrests are in the up position.
- ⚠ **DO NOT** transfer in or out of the Translator on you own unless you have sufficient upper body strength, balance and agility, and have been instructed in transfer techniques by a health professional. Every precaution should be taken to reduce the gap distance between the translator and the target seat or bed. Turn both casters parallel to the object you are transferring to.
- ⚠ **DO NOT** "plop" or drop all body weight suddenly onto the translator seat. Ease gently onto the seat when moving to sit.

Brake Warnings:

- ✓ Check the brakes for proper operation prior to each use. Make sure that all parts are secure and that the moving parts are in good working order.
- ✓ Wheels should roll easily and not wobble.
- ⚠ **DO NOT** use the Translator to provide assistance to the user when rising from a lower position. The Translator may move even if the brakes are in the locked position and result in a fall.
- ⚠ **DO NOT** push the Translator while the brakes are locked. Even if the brakes are locked it may not prevent the Translator from moving if pushed on certain surfaces.

Environmental Warnings:

- ✓ Contact with water or excess moisture may cause parts to rust or corrode.
- ⚠ **DO NOT** use the Translator in a shower, pool or other body of water. Avoid excess moisture (do not leave the Translator in the bathroom while taking a shower).
- ⚠ If the Translator is exposed to extreme temperatures, high humidity and/or becomes wet, check parts are in good working order and that the handgrips are not loose and cannot be twisted. **DO NOT** push the Translator while the brakes are locked. Even if the brakes are locked it may not prevent the Translator from moving if pushed on certain surfaces.
- ⚠ **DO NOT** use or store the Translator near open flames or combustible materials.
- ✓ The surfaces of the Translator can heat up when exposed to direct sunlight or used on hot surfaces that could result in product damage and/or personal injury."
- ⚠ **DO NOT** operate the transport chair on loose, slippery, or uneven surfaces such as sand, gravel, hills, dirt roads, or ice.

Contents

Examine all parts for shipping damage. DO NOT use the translator if there is apparent damage or if any parts (other than the bag) are missing. If there are missing or damaged parts, please call Medline Customer Care toll free at 1-800-MEDLINE.

After receiving the goods, please check the contents for completeness right away.

The content consists of:

- Translator
- Instruction Manual
- Under Seat Bag
- Backrest/Push Bar
- Hardware
 - 2 - 5 Point Knob
 - 2 - Curved Washer
 - 2 - Height Adjustment Screw

Summary

Product overview

Assembly/adjustments

1. Unfolding

- Position the translator in the travel direction.
- Press down on the seat to open the Translator (figure 1). DO NOT use if the translator is not fully open.

CAUTION: Keep hands on the top of the seat and away from any moving parts.

2. Folding

- Hold the translator firmly with one hand on the handle. With your other hand pull up on the pull strap to fold. (figure 2)

3. Attaching the bag

- Unfold the translator and apply the parking brakes.
- Attach the bag by looping the hook and loop straps around the frame and pressing firmly on the bag (figure 3).

4. Adjusting the push handles

- Stand the translator upright and insert the brake handle assembly (figure 4).
- Brake cables must be on the outside of the handle bars and frame.
- Insert the height adjustment screw into the 6 sided hole in the frame. Make sure the screw head recesses into the frame (figure 5).
- Slide the curved washer onto the other end of the screw with the curved side facing the frame.
- Screw on the 5 point knob.
- Repeat steps on the other side. Make sure the push handles are set at the same height and are fully tightened before use.

(Hand tighten only)

5. Adjusting/activating the brakes

Locking Brake

To activate the locking brake, pull both brake handles upward on the push handles. Parking and locking brake act on the fixed wheels (figure 6).

Parking Brake

- Press down on both brake handles until they lock to activate the parking brake (figure 7).
- To release the parking brake pull both brake handles back into middle position (figure 8).

6. Attaching the Backrest/Push Bar

While the translator is unfolded activate the parking brakes. Slide each side of the Backrest/Push Bar into the slots attached to the backside of the translator's handle assembly (figure 9). Press down on the backrest/push bar so the push buttons lock into place. Use ONLY the top bar to push the translator.

Use

Rollator

- Use on a level, firm surface.
- Familiarize yourself with the function of the brakes.
- Place yourself in the translator so your ankles are at the height of the rear wheels.
- Stand with good posture.
- Don't slouch or lean onto the push handles.
- Grasp the push handles with both hands and your arms bent at a slight angle.
- Ensure the footrests are in the UP position prior to each use. Failure to do so can create a tripping hazard.
- When walking, try to stay as close as possible to the push handles. Try to walk in the translator and not behind it. Take small steps and DO NOT push the translator too far ahead.
- Avoid having to lift the translator over obstacles. Use ramps and go around obstacles whenever possible. Use assistance, if avoiding obstacles is not possible.

Transport Chair

- Use for short trips ONLY. The Translator is NOT intended to be used as a full time wheelchair and/or transport chair replacement.
- Make sure the translator rolls easily and that all parts work smoothly.
- Check for noise, vibration, or any change in use. Changes may indicate loose parts or translator damage.
- Check to ensure wheel locks are in good working order.
- Make sure hand grips are not loose and cannot be moved or twisted.
- Use on level, firm surface
- Always assist the occupant in and out of the translator. DO NOT leave the occupant unattended.
- Footrests must be in the up position when attempting to sit or stand.
- Push the translator using ONLY the top bar of the backrest/push bar. DO NOT use the handles to push the user.
- DO NOT attempt to lean forward over the wheels or reach for objects that require changing positions or shifting of weight while seated.

Technical data

Dimensions and weights

Type:	Translator
Height (height of push handles):	35.75 in - 40 in
Seat height:	21 in
Weight:	19.1 lb including bag
Max. load:	300 lb (136 kg)
Max. load of bag:	5 lb (2.2 kg)

Maintenance tips for users

Periodic visual inspection of this product is recommended to ensure that all parts and hardware are secure, that components are in good working order and not cracked, broken or fractured, and that there are no obstructions or impediments to normal, safe operation. If any part or mechanism appears to be unsafe or damaged, do not use the product. If you experience problems, please contact Medline at 1-800-MEDLINE (633-5463) | Medline Canada 1-800-268-2848.

DO NOT modify the transport chair. Examples of modification include: drilling holes, grinding, welding parts, and using non-authorized/non-approved parts. Such modifications may change the transport chair's structure and center of balance, which may increase the risk of serious injury to the user and void the warranty.

1. Cleaning and disinfection

- The frame of the translator can be cleaned with a moist cloth with mild detergent.
- The wheels can be cleaned with a wet synthetic fiber brush (DO NOT use wire brush).
- Push handles, seat and backrests can be wiped down using mild detergent.
- DO NOT use aggressive or caustic detergents to clean the translator.
- Periodically wipe down the translator with a soft cloth and non-abrasive cleaner.
- Dry completely before use.

2. Translator check by the user

2.1. Brake Inspection and Adjustment

- Press the brake shoe to the wheel and tighten the nut on the brake cable just above the spring. Tighten with a few turns, release the brake shoe and apply the brake.
- No further adjustment is needed if the wheel does not spin when the brake is applied.
- If the wheel continues to spin when the brake is applied, tighten the nut further and repeat.
- Repeat on the other brake, if necessary.
- The brake shoe must NOT contact the wheel when the brakes are not applied.

DO NOT use the Translator if the brakes are not functioning properly.

Make sure your brakes are operating properly before EACH use.

If the brakes are not functioning properly, discontinue use immediately and DO NOT use the translator.

2.2. Checking the wheels

Check whether the wheels are intact every time before using.

2.3. Footrests

Check before each use that the footrests are in the up position. If the footrests don't stay up, tighten the bolt attaching the footrests to the frame.

Limited Warranty

Your Medline product is warranted to be free of defects in materials and workmanship for the duration of use by the original purchaser. This limited warranty is an expression of our confidence in the materials and workmanship of our products and offers convenient, dependable service. In the event of a defect covered by this warranty, we will, at our option, repair or replace the device. This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as replaceable components, accessories, casters and grips, which are subject to normal wear and replacement.

Limited Lifetime Warranty on the Frame, 6 months on:

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Some products may not be available for sale in Mexico or Canada. Please contact your Account Representative for additional information. We reserve the right to correct any errors that may occur within this manual. ©2023 Medline Industries, LP. Guardian and Medline are registered trademarks of Medline Industries, LP.