Folding Walker with 5" (12.7 cm) Wheels
Marchette pliante à roulettes de 12,7 cm (5 po)
Andador plegable con ruedas de 5" (12.7 cm)
Blue | Bleu | Azul

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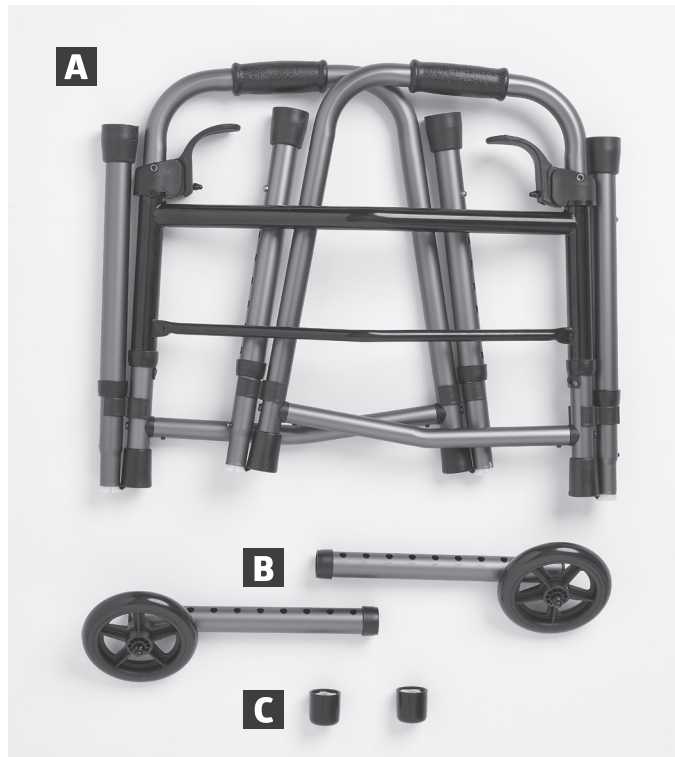
B 2 Wheeled Extensions -
5" (12.7 cm) wheels

C 2 Inserted Glide Caps

(For use with wheels, please replace back rubber tips with included glide caps.)

D Step-By-Step
Assembly Guide

E Safety Guidelines

This guide applies to the following walker:

Item Number	Description	Frame Color	Weight Capacity
MDS86410KDBW	Folding walker with 5" (12.7 cm) wheels	Blue	350 lb (158 kg)

First: Open the Walker

Unfold frame completely ensuring the sides “click” and lock into place.

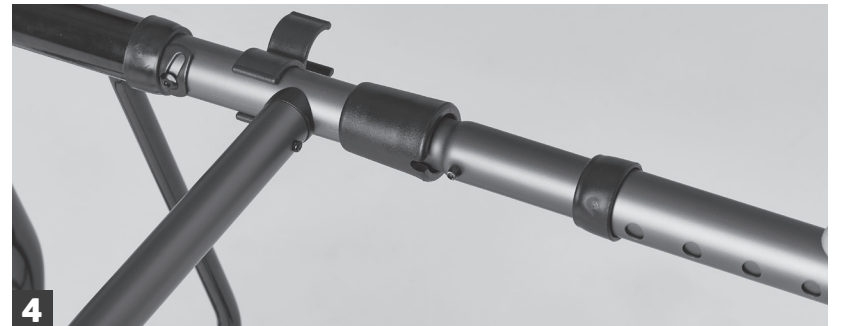
Lay walker on its side.

Next: Attach Front & Rear Legs

Pull front leg out of the clip attached to the frame and insert leg into the bottom of frame.

Inserted leg not secure.

Inserted leg secure. Repeat step 1-3 for all the legs.

NOTE:

Make sure pin is firmly seated into plastic collar slot.

Now: Set Desired Height

Turn walker on its side and depress push-button completely and install leg extension onto leg in desired hole.

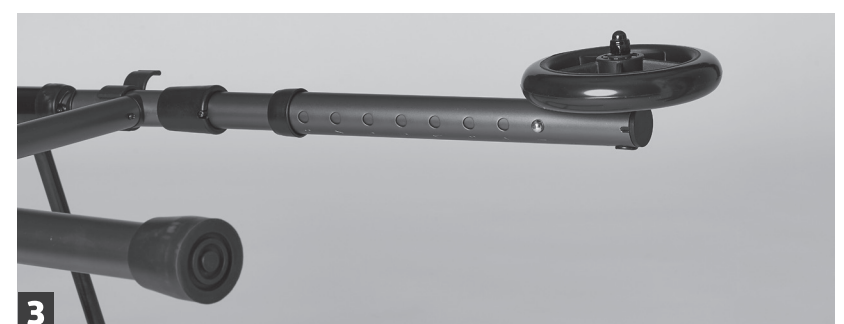
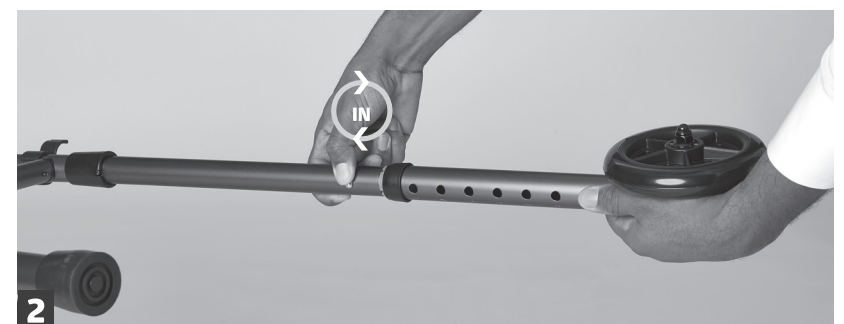
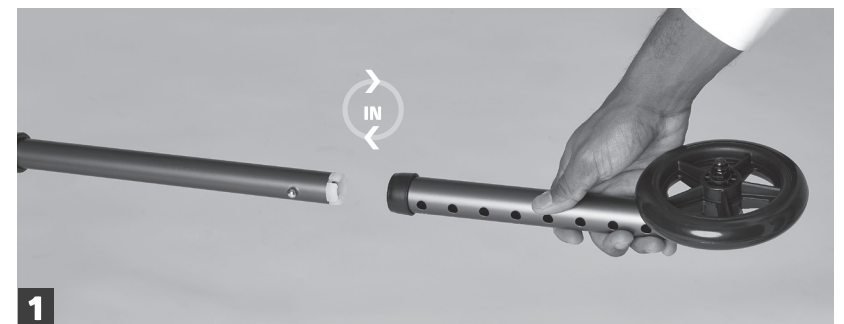
NOTE:

Make sure all 4 leg extensions (with or without wheels) are set at the same overall desired height.

Optional: Attach 5" (12.7 cm) Wheeled-leg Extensions (FRONT LEGS ONLY)

Turn walker on its side and depress push-button completely and install 5" (12.7 cm) wheeled-leg extension onto leg into desired hole. Make sure push-button "clicks" into hole to ensure securing of leg extension to leg. Wheels should be on the outside of the walker.


Push button on wheeled-leg extension is now fully engaged.

Remove back rubber tips from back legs and replace with glide caps.

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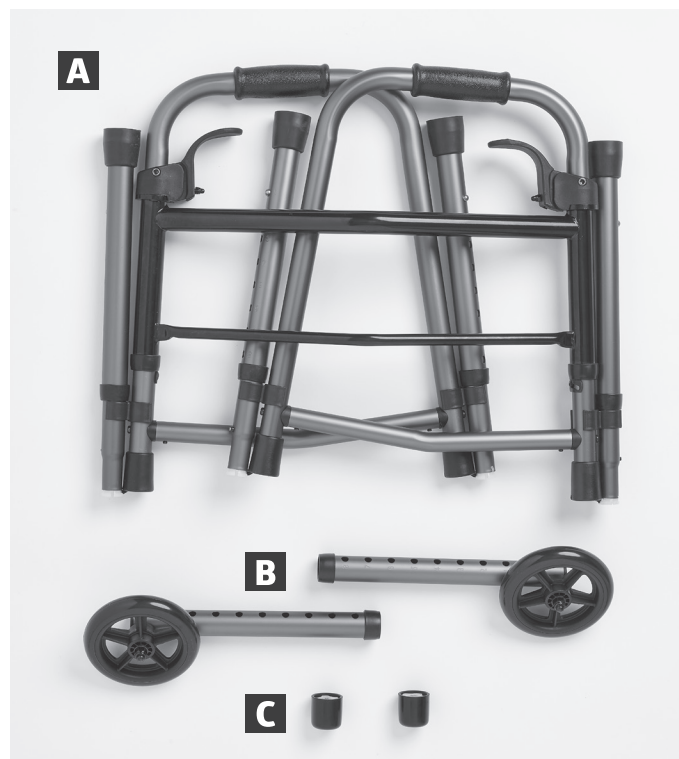Maintenance:

Clean the grips with a mild soap detergent or household cleaner. Do not use an organic solvent such as acetone, MEK, lacquer thinner, lighter fluid, dry cleaning fluid, gasoline or turpentine, that may dissolve the plastic and cause the grip to loosen. The safe working period of this product is five years, providing that it has been used in accordance with the safety instructions, instructions for maintenance and the intended use specified in this manual.

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Medline Canada
1-800-268-2848 | medline.ca | canada@medline.com

Medline Mexico
01-800-831-0898 | medlinemexico.com | mexico@medline.com

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Some products may not be available for sale in Mexico or Canada. We reserve the right to correct any errors that may occur within this brochure. © 2024 Medline Industries, LP. All rights reserved. Medline is a registered trademark of Medline Industries, LP.

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